

smiths medical
surgivet®

Veterinary Equipment and Disposables

Product Catalog



Veterinary
Equipment



Veterinary
Disposables



Caution: US Federal Law restricts these devices to sale by or on the order of a licensed veterinarian.

Contents

Veterinary Equipment	12
 Monitoring Equipment	12
Common Features	12
SurgiVet® V1030 Hand-Held Pulse Oximeter	12
SurgiVet® V3304 Tabletop Pulse Oximeter	13
SurgiVet® V3395 TPR Monitor (Temperature/Pulse/Respiration).....	13
SurgiVet® V6004 Series Non-Invasive Blood Pressure Monitors.....	14
SurgiVet® Hand-Held Capnograph	14
SurgiVet® V9004 Series Capnograph Monitor	15
SurgiVet® Advisor® Vital Signs Monitor	16
SurgiVet® Advisor® 3 Parameter Vital Signs Monitor	17
SurgiVet® Advisor® Veterinary Data Logger	17
Pulse Oximetry Sensors	18
5 Ft Oximetry Extension Cable	18
Replacement Clips	18
ECG Leads and Cables	19
Cardio Companion	19
Optional Accessories	20
Temperature and Respiration Cables	20
Non-Invasive Blood Pressure Cuffs	20
Invasive Pressure Accessories	21
TranStar® Pressure Monitoring System.....	21
TranStar® Mounting Accessories	22
CO ₂ Calibration	22
Agent Calibration.....	22
CO ₂ Accessories.....	23
CO ₂ Monitor Exhaust Kit	23
Sample Lines	23
Airway Adapters.....	24
Printers.....	24
Docking Stations.....	24

Power Supplies.....	25
Printer Paper	25
Battery Eliminator	26
Power Cords	26
Monitor Wall Mounting Options.....	26
Monitor IV Pole Mounting Options.....	26
Roll Stands.....	27
 Infusion.....	28
Medfusion® 3500 Syringe Pump.....	28
Mangum™ Infusion Set.....	28
 Temperature Management	29
HOTLINE® Fluid Warmer	29
SurgiVet® Equator® Convective Warming System.....	29
Accessories.....	29
Level 1® Snuggle Warm® Convective Warming Blankets	30
HOTLINE® Disposable Administration Sets	30
 Anesthesia Equipment.....	31
Common Features of All SurgiVet® CDS 9000 Small Animal Anesthesia Machines.....	31
SurgiVet® Universal CDS 9000 Small Animal Anesthesia Machine Pole Mount.....	31
SurgiVet® Universal CDS 9000 Small Animal Anesthesia Machine Tabletop.....	31
SurgiVet® Universal CDS 9000 Small Animal Anesthesia Machine Conversion Kit.....	32
SurgiVet® LDS 3000 Large Animal Anesthesia Machine.....	32
SurgiVet® Large Animal Anesthesia System.....	32
Common Features of All SurgiVet® Lab/Small Animal Anesthesia Machines	33
SurgiVet® Tabletop Research Anesthesia Machine.....	33
SurgiVet® Multi-Station Lab Animal Anesthesia Machine	33
SurgiVet® Vaporstick Anesthesia Machine	34
V7321 Low Oxygen Pressure Alarm	34

V7320 High Patient Pressure Alarm.....	34
Induction Chambers	34
Pediatric Standard “Y” Circuit	35
Breathing Bags	35
Silicone Tubing	35
Universal F-Circuit	35
Modified Jackson Rees.....	36
C-Pram Breathing Circuit	36
Avian Circuit.....	36
Rodent Circuit Set.....	36
Mayo Tray	36
Sodasorb® LF CO ₂ Absorbent	37
Sodasorb® CO ₂ Absorbent	37
SurgiVet® V7306 Pop-Off Valve	37
SurgiVet® V7309 Pop-Off Occlusion Valve	38
Absorbers	38
Non-Rebreathing Adapter	38
Anesthesia Masks.....	38
Rodent Mask Diaphragms	39
Dual Anesthesia Research Circuit.....	39
Foal Circuit Set	39
“Y” Piece with Luer Fitting	39
Large Animal Endotracheal Tube Funnel Adapter (Silicone)	40
Miscellaneous Adapters	40
 MR Conditional Equipment.....	41
Common Features of SurgiVet® CDS 9000 MR Conditional Small Animal Anesthesia Machines	41
SurgiVet® CDS 9000 MR Conditional Small Animal Anesthesia Machines.....	41
MR Conditional Vaporizers	41
Waste Gas Collection	42
MRI 2550 Small Animal Ventilator	42
Pneupac® Respiratory Ventilator	42

Ventilators	43
SAV 2500 Small Animal Ventilator.....	43
DHV 1000 Large Animal Ventilator	43
Pediatric Bellows.....	44
Foal Bellows	44
Vaporizers	45
Servicing Isoflurane and Sevoflurane Classic T3™ Vaporizers.....	45
Cleaning and Calibration	45
Vaporizer Exchange Program.....	45
SurgiVet® Classic T3™ Vaporizer	45
Vaporizer Accessories	46
Anesthetic Key Filler	46
Anesthetic Anti-Spill Adapter.....	46
Gas Delivery Management.....	47
Hard Line Assemblies (Wall Outlets)	47
Recessed Style Wall Outlet.....	47
Oxygen Manifolds	47
Soft Line Assemblies (Hose Assemblies).....	48
Pre-Assembled Hoses.....	48
Hose Assemblies	48
Gas Hoses	48
Air Hose Connectors (Yellow)	48
Carbon Dioxide Hose Connectors	49
Evacuation Hose Connectors (Purple)	49
Miscellaneous Hose Connectors.....	49
Nitrous Hose Connectors (Blue)	50
Vacuum Hose Connectors	50
Oxygen Hose Connectors With Barbs (Green)	51
Oxygen Hose Connectors (Green)	52
E-Tank Accessories	53
H-Tank Accessories.....	53
Universal Double E-Tank Manifold.....	53

E-Tank Stabilizer	53
E-Tank Holders and Regulator.....	54
E-Tank Accessories	54
Respiratory Flowmeters.....	54
Flowmeter Replacement Parts	55
Humidifier System.....	55
Demand Valves	55
 Waste Gas Management	56
Anesthesia Detection Badge	56
SurgiVet® AES Gas Evacuation System	56
Waste Gas Evacuation Accessories.....	56
SurgiVet® Enviro-Pure® Filter Canister.....	57
Scavenger Interface Valve For Active Systems	57
SurgiVet® Lab Animal Evacuation System	57
Evacuation Charcoal.....	58
 Rescue and Resuscitation	59
Resuscitation Bags.....	59
Recovery Oxygen Masks	59
Replacement Mask Diaphragms.....	59
Veterinary Disposables.....	12
 Airway.....	60
Non-Cuffed Endotracheal Tube (Silicone).....	60
Non-Cuffed Wire Reinforced Endotracheal Tube (Silicone).....	60
Cuffed Wire Reinforced Endotracheal Tubes (Silicone)	61
Cuffed Endotracheal Tubes (Silicone)	61
Disposable Laryngoscopes.....	62
PressureEasy® Cuff Pressure Controller.....	62
Endotracheal Tube Connectors.....	63
Large Animal Endotracheal Tube Funnel Adapter (Silicone)	63
Endotracheal Tube Replacement Accessories (Silicone)	64
AIRCARE® PVC Endotracheal Tubes	64

Nasotracheal and Tracheostomy Tubes.....	65
Foal Nasal/Porcine Endotracheal Tube (Silicone)	65
Equine Post Anesthetic Nasal Tube (Silicone)	65
Tracheostomy Tubes (Silicone).....	65
 Gastroenterology.....	66
Esophageal Tubes.....	66
Esophagostomy Tube Sets	66
Van Noort Esophagostomy Tube Sets	66
Feline Esophagostomy Tubes (Silicone).....	66
Oral/Nasal Oxygen/Feeding Tubes.....	67
Economy ClearView Oral/Nasal Oxygen/Feeding Tubes.....	67
Stomach Tubes	68
Gastrostomy Tube Introduction Set.....	68
Feline Orogastric Feeding Tube	68
Feline Mouth Gag.....	68
Peg Feeding Tubes	69
Fixation Disc (Sterile)	69
 Fluid Administration and Therapy	70
Silicone Stomach Tubes	70
Connecting Tube	70
Bag to Bag Connectors.....	70
Wide Bore Infusion Set (No Chamber)	71
Stainless Steel Spike Adapter	71
Subcutaneous Catheter Sets.....	71
Peripheral IV Catheters.....	72
Jelco® IV Catheters.....	72
OPTIVA® IV Catheters	72
Blood Collection	73
Injection Caps and T-connector Extension Sets	73
Saf-T Wing® Blood Collection & Infusion Set.....	73
Injection Caps	73
Gravity Fluid Administration Sets	74

Pressure Infusion	74
C-Fusor® Pressure Infusor.....	74
Clear-Cuff® Pressure Infusor.....	74
T-connector Extension Sets	75
Extension Sets	75
 Needles	76
Vet-Core™ Biopsy Needles	76
Needles with Peel-Off Sheaths	76
Equine Sacral Joint Needle	76
 Reproduction.....	77
Equine Uterine Flushing Catheter	77
Equine Fluid Delivery System.....	77
 Urology	78
Slippery Sam™ Tomcat Urethral Catheters	78
Tomcat Urethral Catheters (Silicone).....	78
Little Herbert® Luer Lock Adapter.....	78
Economy ClearView Foley Catheters (Silicone)	79
Premium Foley Catheters (Silicone)	79
 Central Venous Catheters.....	80
Central Venous Seldinger Catheter Sets	80
Central Venous Peel-off sheath catheter sets.....	80
 Wound Management	81
Chest Drains	81
Chest Drainage Tubes	81
Multi-Purpose Drainage Tube*	81
Pneumothorax Sets.....	81
Heimlich One-Way Valve.....	82
Thoracic Drainage Catheter	82
Two-Part Drainage Catheter Sets.....	82
Drainage Catheters and Tubes.....	83
Pericardiocentesis Catheter Set	83

Pericardiocentesis Needle & Catheter	83
Broncho-Alveolar Lavage Catheter (Silicone).....	83
Canine Broncho-Alveolar Lavage Catheter	84
Guttural Pouch Endoscopic Flushing Catheter.....	84
Peritoneal Lavage Catheter.....	84
Wound Drains	85
Continuous Suction Bulbs	85
Fenestrated Catheter Wound Drain	85
Modified Penrose Drains (Silicone)	85
Penrose Drains (Silicone).....	85
Closed Wound Vacuum Drains (Silicone)	86
Flexi-Drain Multi-Purpose Wound Drain (Silicone)	86
 Consumable Product Accessories	87
Stopcocks (Sterile).....	87
Multi-Purpose Tubing Adapters (Sterile)	87
Luer Lock Plug (Sterile)	88
Clamps (Sterile).....	88
Silicone Adhesive (Non-Sterile)	88
Fixation Disc (Sterile)	88
Wire Guides, Stylets, Introduction Tubes	89
Endotracheal Tube Stylet With Stopper	89
ClearView PTFE Coated Wire Guide	89
Wire Guide Stiffening Stylets.....	89
Metal Stiffening Stylets	90
Stainless Steel Threaded Wire Guides	90
Introduction Tubes.....	90
 Index.....	91
Out of Warranty Services (per return)	96
Equipment Warranty Extension Program (per device, per year)	96
Notes.....	97

VETERINARY EQUIPMENT

Monitoring Equipment

Common Features

PULSE OXIMETRY MONITORS

- Digital pulse oximetry with SAC technology perfect for low perfusion and/or higher motion situation



V1030

ORDERING INFORMATION

Product Codes	Description
V1030	Hand-held Pulse Oximeter

OPTIONAL POWER SUPPLY CHOICES FOR V1030 - NOT INCLUDED IN PURCHASE

- WW1090 Custom lithium ion rechargeable battery (30 hrs continuous use; 3 hr recharge)
- WW1095 Power supply universal with US/ Japan adapter 30 watt, 100-240V, 50Hz
- WW1095OUS Power supply universal with international blades 30 watt, 100-240V, 50Hz
- WW1096SYS AC power and rechargeable lithium ion battery system
- WW1096OUS AC power and rechargeable lithium ion battery system (OUS)s

ALL MONITORS

- Veterinary specific software
- Manuals and quick reference cards available at www.surgivet.com

SurgiVet® V1030 Hand-Held Pulse Oximeter

FEATURES

- Low perfusion performance
- Motion tolerance
- Pulse Amplitude Index (PI) bargraph indicator to help identify and maintain a good test site
- Memory storage for up to 99 patients
- Quick selection of default alarm limit values by patient type: cat, dog, horse, factory and custom settings
- Audible and visual alarms
- Multiple power choices (see p. 13)
- 3 year warranty
- Dimensions: 6.1" H x 3.3" W x 1.7" D (15.5 cm H x 8.4 cm W x 4.3 cm D)
- Weight: 12 oz. (340 grams)

ITEMS INCLUDED WITH EACH

- 3311 5 ft. oximetry extension cable
- V1700 reflectance oximetry sensor
- V1703 universal "Y" oximetry sensor
- Four (4) "AA" (LR6) alkaline batteries
- WW1018B blue protective glove
- A printed quick reference guide is also included
- Optional docking station provides option to charge a second lithium ion battery (optional)

OPTIONAL ITEMS NOT INCLUDED

- WW1025SYS docking station
- WW1025OUS docking station (OUS)
- WW1028SYS docking station and printing station
- WW1028OUS docking station and printing station (OUS)
- WW1027 printing paper
- WW1098 universal mounting bracket
- WW1098C (UMB) with IV pole clamp
- WW1026SYS attachable thermal printer



V3304

ORDERING INFORMATION

Product Codes	Description
V3304	Tabletop Pulse Oximeter

SurgiVet® V3304 Tabletop Pulse Oximeter**FEATURES**

- Digital SpO₂ and heart rate
- Adjustable alarm limits
- Visual and audible alarm signals
- 99 patient memory
- Ability to transfer real time data or trend data via serial cable connections
- Dimensions: 3.24" H x 8.5" W x 5.5" D (21.6 cm H x 8.2 cm W x 14 cm D)
- Weight: 30 oz. (850 grams)
- Optional rollstand (3388)
- Optional pole mount bracket (WW3356)

ITEMS INCLUDED WITH EACH

- 1613 universal power supply with US plug AC/DC power supply 100-240VAC/12DC (US, Japan)
- 1613EU universal power supply with EU plug AC/DC power supply 100-240VAC/12DC (Euro) (optional)
- 1613UK universal power supply with uk plug AC/DC power supply 100-240VAC/12DC (UK) (optional)
- 1613AU universal power supply with AU plug AC/DC power supply 100-240VAC/12DC (Australia) (optional)
- V1700 reflectance oximetry sensor
- V1703 universal "Y" oximetry sensor
- 3311 5 ft oximetry extension cable



V3395

ORDERING INFORMATION

Product Codes	Description
V3395	TPR Monitor

SurgiVet® V3395 TPR Monitor (Temperature/Pulse/Respiration)**FEATURES**

- Digital SpO₂, heart rate, temperature, and respiration
- Plethysmographic waveform display
- Adjustable alarm limits
- Visual and audible alarm signals
- Optional infrared printer (8411)
- Optional reusable esophageal/rectal temperature probe (WWW3418)
- Dimensions: 5.0" H x 4.38" W x 2.9" D (12.2 cm H x 11.1 cm W x 7.37 cm D)
- Weight: 22 oz. (0.63 kg)
- Optional pole mount bracket (8409)

ITEMS INCLUDED WITH EACH

- V3396 power supply 120 VAC 9 VDC 1 A
- V3397 power supply 220 VAC 9 VDC 1 A (Optional) (OUS)
- V1703 universal "Y" sensor
- V1700 reflectance sensor
- 3311 5 ft oximetry extension cable
- V3413 temperature cable
- V3417 temperature probe disposable
- V3422 respiration sensor
- WW1100 airway adapter, straight
- 20690B3 protective boot



V60046

ORDERING INFORMATION

Product Codes	Description
V60040	NIBP
V60041	NIBP and Digital SpO ₂
V60045	NIBP and Built-In Printer
V60046	NIBP, Digital SpO ₂ , and Built-In Printer

SurgiVet® V6004 Series Non-Invasive Blood Pressure Monitors

FEATURES

- Oscillometric non-invasive blood pressure technology
- Systolic, diastolic, and mean NIBP values
- Four available cuff sizes will comfortably fit a variety of patients
- Two modes of use: SA mode for patients under 20 lbs and LA mode for patients over 20 lbs
- Ability to transfer real time data or trend data via serial cable connections
- Built-in printer with one roll of paper (V60045 & V60046 only)
- Base unit dimensions: 3.24" H x 8.5" W x 5.5" D (8.2 cm H x 21.6 cm W x 14.0 cm D)
- Base unit weight: 3.5 lbs. (1.6 kg)
- Optional rollstand (3387)
- Optional pole mount bracket (WW3356)
- Paper refill 4-pack (6012)

ITEMS INCLUDED WITH V6004 MONITOR

- 1616 universal power supply with US plug AC/DC power supply 100-240vac/12dc (US, Japan)
- 1616EU universal power supply with EU plug AC/DC power supply 100-240VAC/12DC (EURO) (optional)
- 1616UK universal power supply with UK plug AC/DC power supply 100-240VAC/12DC (UK) (optional)
- 1616AU universal power supply with AU plug AC/DC power supply 100-240VAC/12DC (Australia) (optional)
- V1703 universal "Y" sensor (V60041 & V60046 only)
- V1700 reflectance sensor (V60041 & V60046 Only)
- 3311 5 ft oximetry extension cable (V60041 & V60046 Only)
- V6148SA NIBP cuff kit



V8401

RECOMMENDED ITEMS FOR MAINTENANCE

- 8217 CO₂ calibration kit (5093, 8061, 8211, 8223 included)
- 8208 10 gas manifold filters

OPTIONAL ITEMS NOT INCLUDED

- 8404 battery charger 100-240 VAC, 9 VDC, 1.7 mA (for use with 8408 only)
- 8408 rechargeable battery (for use with 8404 or 8404A only)

SurgiVet® Hand-Held Capnograph

ORDERING INFORMATION

Product Codes	Description
V8401	Hand-Held Capnograph
V8400D	Hand-Held Capnograph w/ SpO ₂ Digital Pulse Oximetry

FEATURES

- 21-0464-01 battery eliminator 105-125 VAC, 90 Hz (US Port)
- 21-0465-10 battery eliminator (AU)
- 21-0466-50 battery eliminator (EU)
- 21-0467-20 battery eliminator (UK)
- 20690B2 protective boot (purple)
- V1175 CO₂ monitor exhaust kit
- WW8214 patient attachment kit includes:
 - 8208 10-gas manifold filters
 - 8211 10-4 ft sample lines
 - WW1100 10-straight
 - Airway adapters
 - Operations manual
 - V1700 reflectance oximetry sensor (V8400D only)
 - V1703 universal "Y" sensor (V8400D only)

ITEMS INCLUDED WITH V6004 MONITOR

- Sidestream technology for application with intubated and non-intubated patients
- End tidal CO₂ numeric value, end tidal CO₂ waveform and respiration rate
- Visual and audible alarm signals
- Adjustable alarm limits
- Optional infrared printer (8411)
- Dimensions: 5.0" H x 4.38" W x 2.9" D (12.2 cm H x 11.1 cm W x 7.37 cm D)
- Weight: 22 oz. (0.63 kg)
- Optional pole mount bracket (8409)
- Optional battery charger 100-240 VAC 9 VDC 1.7 mA (8404)
- Optional battery (8408)

SurgiVet® V9004 Series Capnograph Monitor



V90041

ORDERING INFORMATION

Product Codes	Description
V90040	Capnograph
V90041	Capnograph and Digital Pulse Oximetry
V90043	Capnograph, Digital Pulse Oximetry, and FiO ₂ (Fractionally Inspired O ₂)
V90041LF	Low Flow Capnograph and Digital Pulse Oximetry

FEATURES

- Incorporates sidestream technology for application with intubated and non-intubated patients
- End tidal CO₂ numeric value, end tidal CO₂ waveform, inspired CO₂, and respiration rate
- Ability to transfer real time data or trend data via serial cable connections
- Dimensions: 3.5" H x 10" W x 5.5" D (8 cm H x 25.4 cm W x 13.9 cm D)
- Weight: 4.8 pounds (2.2 kg)
- Optional rollstand (3388)
- Optional pole mount bracket (WW3356)

RECOMMENDED ITEMS FOR MAINTENANCE

- 8217 CO₂ calibration kit (Includes: 5093, 8061, 8211, 8223)
- 1178N 5-aquaknot moisture traps
- 9048 2-13 mm CO₂ disk filters
- 1187 aquaknot adapter

ITEMS INCLUDED WITH V6004 MONITOR

- 1616 universal power supply with US plug AC/ DC power supply 100- 240VAC/24 volt (US, Japan)
- 1616EU universal power supply with EU plug AC/DC power supply 100-240VAC/12DC (EURO) (optional)
- 1616UK universal power supply with UK plug AC/DC power supply 100-240VAC/12DC (UK) (optional)
- 1616AU universal power supply with AU plug AC/DC power supply 100-240VAC/12DC (Australia) (optional)
- V1700 reflectance sensor (V90041 & V90043 Only)
- V1703 universal "Y" sensor (V90041 & V90043 only)
- 3311 5 ft oximetry extension cable (V90041 & V90043 only)
- V1175 CO₂ monitor exhaust kit
- 9193T FiO₂ kit (V90043 only)
- Start-up kit includes:
 - WW1100 6-straight airway adapters
 - V1186 3-nasal sample lines
 - 1178N 2-aquaknot moisture traps
 - 8044 6-8 ft sample lines
 - V1182 2-3.5 mm dead space adapters
 - 9048 1-13 mm CO₂ disk filter

SurgiVet® Advisor® Vital Signs Monitor



V9204

ORDERING INFORMATION

Product Codes	Description
V9200	3 lead ECG, digital SpO ₂ , heart rate, and NIBP
V9200IBP/Temp	3 lead ECG, digital SpO ₂ , heart rate, NIBP, IBP, and temperature
V9200P	3 lead ECG, digital SpO ₂ , heart rate, NIBP, and built-in printer
V9201	3 lead ECG, digital SpO ₂ , heart rate, NIBP, end tidal CO ₂ , and respiration
V9201P	3 lead ECG, digital SpO ₂ , heart rate, NIBP, end tidal CO ₂ , respiration, and built-in printer
V9203	3 lead ECG, digital SpO ₂ , heart rate, NIBP, 2-channel IBP, end tidal CO ₂ , 2-channel temperature, respiration
V9204	3 lead ECG, digital SpO ₂ , heart rate, NIBP, 2-channel IBP, end tidal CO ₂ , 2-channel temperature, respiration, built-in printer

FEATURES

- Four waveform color display screen
- Intuitive point and click user-interface
- Ability to view all parameters trending information on screen
- Ability to transfer real time or trend data via serial cable connections
- Optional veterinary data logger
- Two year warranty
- Optional roll stand (3387)
- Optional wall mount (3393)
- Optional CDS9000 monitor mounting bracket (WWW90MB)

DIMENSIONS:

- 12" H x 13.25" W x 7" D (30.5 cm H x 34 cm W x 18 cm D)
- Weight: 9-11 lbs. (4.1-5 kg)

ADDITIONAL ITEMS INCLUDED WITH V9200 IBP/TEMP

- V3413 temperature cable
- V3417 temperature probe (disposable)
- V6401 transducer cable
- V6402 invasive pressure transducer

ADDITIONAL ITEMS INCLUDED WITH V9203 & V9204 MODELS

- V3417 temperature probe
- V3413 temperature cable
- WWW3418 reusable esophageal/rectal temperature probe (optional)
- V6401 IBP transducer cable
- V6402 invasive pressure transducer
- Start-Up Kit Includes:
 - 8075 Moisture traps
 - 8044 6-8 ft sample lines (V9201 also)
 - V1186 3-nasal sample lines (V9201 also)
 - WW1100 6-straight airway adapters (V9201 also)
 - V1175 CO₂ monitor exhaust kit (V9201 also)
 - 9048 1-13 mm CO₂ disk filter
 - V1182 1-3.5 mm dead space adapters

RECOMMENDED ITEMS FOR MAINTENANCE

- 8217 CO₂ calibration kit
- (5093, 8061, 8211, 8223)
- 1179 2-CO₂ absorber
- 9048 2-13 mm CO₂ disk filter
- 8075 1-moisture trap

ITEMS INCLUDED WITH THESE ADVISOR® MODELS

- WW3005 power cord 110 V
- 3005A power cord 2 prong-europe (optional)
- V1700 reflectance sensor
- V1703 universal "Y" sensor
- 3311 oximetry extension cable (5 ft.)
- V3110TL/V3110TLS 3 lead ECG
- V6148SA NIBP cuff kit
- 6111 printer paper (if a printer is ordered)



WWV9230

ORDERING INFORMATION

Product Codes	Description
WWV9230	Advisor® 3 Parameter Monitor

*Not compatible with Data Logger

SurgiVet® Advisor® 3 Parameter Vital Signs Monitor

The Advisor® multi-parameter vital signs monitor is also available in a basic three parameter configuration, providing veterinary specific vital signs measurements for non-invasive blood pressure, pulse oximetry, and ECG. With its large easy to read display and intuitive user interface, this configuration of the Advisor® vital signs monitor is ideal for the budget conscious buyer, offering the same high quality and reliability you've come to rely on from the SurgiVet® Advisor® vital signs monitor product family. This monitor cannot be upgraded.

FEATURES

- Pulse oximetry with pulse rate
- Non-invasive blood pressure
- ECG with heart rate
- 10.4" LCD with color monitor display
- 2 year warranty on monitor
- Dimensions: 12" H x 7" W x 13.25" D (30.5 cm H x 18 cm W x 34 cm D)
- Weight: 9-11 lbs. (4.1-5 kg)
- Optional roll stand (3387)
- Optional wall mount (3393)
- Optional CDS9000 monitor mounting bracket (WWV90MB)

ITEMS INCLUDED WITH THESE ADVISOR® MODELS

- V1703 universal "Y" sensor
- 3311 oximetry extension cable
- V3110TL 3-lead ECG cable
- V3110TLS ECG lead wires
- 31543B2 medium NIBP cuff (5-15 cm inflation size)
- 31544B1 6' NIBP supply hose
- Power cord



WWV9220 as part of the 21-6319-0100-01 kit

SurgiVet® Advisor® Veterinary Data Logger**ORDERING INFORMATION**

Product Codes	Description
21-6319-0100-01*	Advisor® Veterinary Data Logger and Software Kit (Microsoft® Excel® 2007, 2010 and 2013)

*Not compatible with WWV9230

FEATURES

- Record real time trend data at regular intervals
- Capture a 30-second primary ECG waveform image
- Capture a screen image
- Upload data to a PC (not MACs); supports various formats (pdf, xls)
- Supports Microsoft® Excel®
- Choice of basic or detailed anesthesia record
- SD (Secure Digital) flash card and card reader included (2 gb)
- Dimensions: .89" H x 2.01" W x 2.30" D (22.5 mm H x 51.1 mm W x 58.5 mm D)
- Weight: 38 grams (1.34 oz)



V1700



V1703



V1707



3311

Pulse Oximetry Sensors

Pulse oximetry sensors work exceptionally well on pink perfused tissue such as a patient's tongue, toe webbing, vulva, prepuce, or ventral tail base.

ORDERING INFORMATION

Product Codes	Description	Features
V1700	Reflectance Sensor	The reflectance sensor can be applied to the ventral tail base of a patient and secured with a cohesive bandage. For cats and dogs, if the sensor is inserted rectally the red light should be shining on the sphincter muscle.
V1703	Universal "Y" Sensor	The universal "Y" sensor is designed to be used on areas of thicker tissue. It is the standard clip used in most small animal practices.
V1707	"C" Sensor	The "C" sensor is perfect for cats, dogs, and horses. The sensor should be placed on the thigh, metatarsal, or metacarpal for smaller patients. The sensor should be placed over the achilles tendon, tongue, prepuce, vulva, or through the toe webbing on large patients.
V3078	Mini Clip Sensor	The mini clip sensor can be used on a variety of animals ranging in size, but is ideal for small dogs, cats and some exotics.

Replacement Clips

ORDERING INFORMATION

Product Codes	Description	Features
V1709	10 "C" clip replacement	For use with the V1707
V1720	1 universal "Y" clip replacement	For use with the V1703

5 Ft Oximetry Extension Cable

ORDERING INFORMATION

Product Codes	Description	Note
3311	5 ft oximetry extension cable	Provides the ability to use pulse oximeter sensors at farther distances from monitors
3311L	15 ft oximetry extension cable	Provides the ability to use pulse oximeter sensors at farther distances from monitors



V3110TLS

ECG Leads and Cables

For use with V3404, V3404P, and Advisor® vital signs monitor.

ORDERING INFORMATION

Product Codes	Description
V3110TLS	3 lead ECG wires with alligator clips, 40"
V3110TLL	3 lead ECG wires with alligator clips, 60"
V3110TL	3 lead ECG cable (can be used with V3110TLS, V3110TLL)
V3406FL	5 lead ECG cable (can be used with V3110TLS, V3110TLL, V3406FLS, 3107)
V3406FLS	5 lead ECG wires with alligator clips, 40"
21-0460-51	ECG cable 3-lead shielded SNAP
21-0461-51	ECG wires 3-lead 40" shielded SNAP
21-0462-51	ECG cable 5-lead shielded SNAP
21-0463-51	ECG wires 5-lead 40" shielded SNAP



Cardio Companion

Cardio Companion

The Cardio Companion is a device used to monitor core body temperature and ECG via an esophageal lead. Compatible with most ECG lead wires. Works with the Advisor® vital signs monitor and the V3404 series (except temp).

ORDERING INFORMATION

Product Codes	Description
V706000	Control box
32491B2	Small green esophageal lead
32491B3	Medium blue esophageal lead
V706001	Cardio companion with small and medium lead
V706004	Cardio companion with two small leads

Temperature and Respiration Cables

For use with V3395 and Advisor® vital signs monitor.

ORDERING INFORMATION

Product Codes	Description
V3413	Temperature cable
V3417	Temperature probe (disposable)
WWW3418	Reusable esophageal/rectal temperature probe
V3422	Respiratory cable and sensor (V3404P and V3395 Only)



V3413



V3417



WWW3418



V3422

Non-Invasive Blood Pressure Cuffs

Fits patients comfortably because of a 360° bladder that eliminates pinching. The 360° bladder also eliminates the need to align an arrow/mark over an artery. The durable cuffs can withstand 10,000 inflations to 200 mmHg and are machine washable. A reinforced closure system ensures that the cuffs will stay on and can handle wear and tear.

ORDERING INFORMATION

Product Codes	Description	Size
31543B1	Small NIBP cuff	3-9 cm
31543B2	Medium NIBP cuff	5-15 cm
31543B3	Large NIBP cuff	9-25 cm
31543B4	Extra large NIBP cuff	17-45 cm
31544B1	6 ft hose	
31545B1	Cap	
V6148SA	NIBP cuff kit (includes one of each listed above)	



V6148SA

Optional Accessories

ORDERING INFORMATION

Product Codes	Description
31544B2	3 ft hose
31544B3	10 ft hose



V6402

Invasive Pressure Accessories

For use with Advisor® vital signs monitor.

ORDERING INFORMATION

Product Codes	Description
V6401	Transducer cable
V6402	Invasive pressure transducer








TranStar® Pressure Monitoring System


TranStar® Pressure Monitoring System

The TranStar® pressure monitoring system comprises a series of intelligently designed products built with high quality materials under stringent quality control.


- Accurate to AAMI specifications and industry standards
- Economical without compromise, using quality Smiths Medical components
- Dependable and simple priming with straight fluid path
- Ergonomic trigger flush device
- Modular transducer mounting plate set-up
- Transducer chip is integrated into the patient disposable

ORDERING INFORMATION

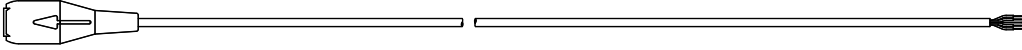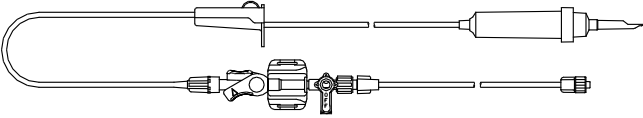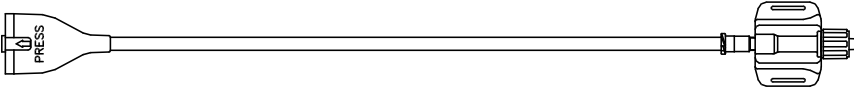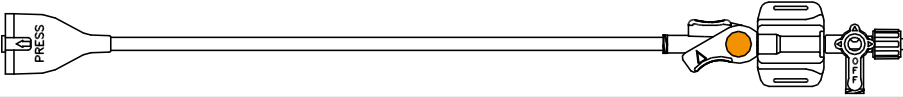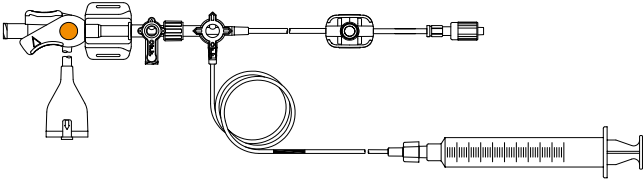Product Codes	Description	Units per case
MX95114	TranStar® pressure transducer cable	1
		
MX9501T	TranStar® 12" (29 cm) patient mount monitoring kit	10
		
MX950	TranStar® disposable pressure transducer with 12" cable	10
		
MX950SCNT	Neonatal TranStar® disposable pressure transducer with stopcock and 30 cc/hr flush with orange insert	10
		
MX9543	TranStar® kids kit™ 19" (48 cm) closed blood sample set	10
		

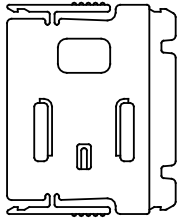

MX800

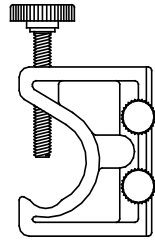

MX240

TranStar® Mounting Accessories

ORDERING INFORMATION

Product Codes	Description	Units per case
MX800	TranStar® modular transducer mounting plates	20
MX240	TranStar® pole clamp for mounting a transducer plate (fits 3/4" to 1 1/2" IV pole)	1



8217

CO₂ Calibration

ORDERING INFORMATION

Product Codes	Description
8217	CO ₂ calibration kit: 19 liter tank of calibration gas, regulator, and sample line adapter (For use with V9004, V8401, and Advisor® vital signs monitor)
5093	CO ₂ calibration gas canister only
8061	Calibration gas regulator only
8223	Calibration adapter (V9004 series)

Agent Calibration

ORDERING INFORMATION

Product Codes	Description
1502	Agent calibration gas canister (V9400 monitor only)
8061	Calibration gas regulator only
8223	Calibration adapter

CO₂ Accessories

ORDERING INFORMATION

Product Codes	Description
1178N	(5) Aquaknot moisture traps
1187	Aquaknot adapter (V9004 and V9400 monitors)
1179	(2) CO ₂ absorber (Advisor® vital signs monitor)
8208	10-gas manifold Filters (V8401 monitor)
8075	Moisture trap (Advisor® vital signs monitor)
9048	(2) CO ₂ disk filter 13 mm (rear filter for V9004 series and Advisor® vital signs monitors)
9193T	FiO ₂ sensor kit: (1) 9190 FiO ₂ sensor, 1-9189 adapter "T", and (1) 9191 cable (V90043 monitor)



1178N



1187

1179



8208

8075



9048

9193T

CO₂ Monitor Exhaust Kit

ORDERING INFORMATION

Product Codes	Description
V1175	This kit will allow waste gas scavenging for CO ₂ monitors. Can be used with the Advisor® vital signs monitor, V8401 and V9004 series.



V1175

Sample Lines

ORDERING INFORMATION

Product Codes	Description
V1186	Nasal sample lines (qty 6)
8211	4 ft sample lines (qty 10)
8044	8 ft sample lines (qty 10)
WW1140	15 ft extension sample lines (qty 10)



V1186

8211



WW1100

Airway Adapters

For use with CO₂ and respiration monitors.

ORDERING INFORMATION

Product Codes	Description
WW1100	Straight airway adapters 13 ID x 15 OD mm (qty 10)
1151	Pediatric airway adapters 13 ID x 15 OD mm (qty 10)
V1180	2.5 mm dead space adapters (sold individually)
V1181	3.0 mm dead space adapters (sold individually)
V1182	3.5 mm dead space adapters (sold individually)



1151



V1180



8411

Printers

ORDERING INFORMATION

Product Codes	Description
8411	Infrared printer (monitors: V3395, V8401 and V8400D) Power supply not included
WW1026SYS	Attachable thermal printer (attaches to the docking station) (monitor: V1030 - not pictured)



WW1025SYS

Docking Stations

ORDERING INFORMATION

Product Codes	Description
WW1025SYS	Docking station
WW1025OUS	Docking station (OUS)
WW1028SYS	Docking station and printer system
WW1028OUS	Docking station and printer system (OUS)



WW1027

Printer Paper

ORDERING INFORMATION

Product Codes	Description
3409	4 Pack (printer: V3408)
6012	4 Pack (monitors: V60045, V60046)
8416	4 Pack (printer: 8411)
6111	4 Pack (Advisor® vital signs monitor)
WW1027	Printer paper, 5 pack (printer: WW1026SYS, used with monitor: V1030)

Power Supplies

ORDERING INFORMATION

Product Codes	Description
1613	Universal power supply with US plug AC/DC power supply 100-240VAC/12DC (US, Japan) (V3304)
1613EU	Universal power supply with EU plug AC/DC Power supply 100-240VAC/12DC (EURO) (V3304)
1613UK	Universal power supply with UK plug AC/DC power supply 100-240VAC/12DC (UK) (V3304)
1613AU	Universal power supply with AU plug AC/DC power supply 100-240VAC/12DC (Australia) (V3304)
1616	Universal power supply with US plug AC/DC power supply 100-240VAC/12DC (US, Japan) (V6004 and V9004 series)
1616EU	Universal power supply with EU plug AC/DC power supply 100-240VAC/12DC (EURO) (V6004 and V9004 series)
1616UK	Universal power supply with UK plug AC/DC power supply 100-240VAC/12DC (UK) (V6004 and V9004 series)
1616AU	Universal power supply with AU plug AC/DC power supply 100-240VAC/12DC (Australia) (V6004 and V9004 series)
WW1090	Custom lithium Ion rechargeable battery (30 hrs continuous use; 3hr recharge†) (V1030)
WW1095	Power supply universal with US/Japan adapter 30 watt, 100-240V, 50Hz (V1030)
WW1095OUS	Power supply universal with international blades 30 watt, 100-240V, 50Hz (V1030)
WW1096SYS	AC power and rechargeable lithium ion battery system (V1030)
WW1096OUS	AC power and rechargeable lithium ion battery system (OUS) (V1030)
21-0468-01	Power supply 208-252 VAC, 13.2 VAC, 1 A (monitor: V3408)
V3396	Power supply 120 VAC, 9 VDC, 1 A (monitor: V3395)
V3397	Power supply 220 V, 9 VDC, 1 A (monitor: V3395)
8404	Battery charger 100-240 VAC, 9 VDC, 1.7 mA (monitor: V8401) (for use with 8408 only)
8408	Rechargeable battery (monitor: V8401) (for use with 8404 or 8404A only)



21-0464-01

Battery Eliminator

Only for use with V8401 and V8401D

ORDERING INFORMATION

Product Codes	Description
21-0464-01	Battery eliminator 105-125 VAC, 90 Hz (US Port)
21-0465-10	Battery eliminator (AU)
21-0466-50	Battery eliminator (EU)
21-0467-20	Battery eliminator (UK)



WW3005

Power Cords

ORDERING INFORMATION

Product Codes	Description
WW3005	Power cord 110 V (monitor: Advisor® vital signs monitor)
3005A	Power cord 2 prong-europe (monitor: Advisor® vital signs monitor)



3393

Monitor Wall Mounting Options

For use in areas where space is limited.

ORDERING INFORMATION

Product Codes	Description
3393	Advisor® vital signs monitor wall mount
WW1098	Universal mounting bracket (monitor: V1030)



8409

Monitor IV Pole Mounting Options

Allows a monitor to be mounted to an IV pole.

ORDERING INFORMATION

Product Codes	Description
WW3356	IV pole bracket (monitor: V3304, V6004 and V9004 series)
8409	Pole mount bracket (monitor: V8401, V3395)
WW1098C	Universal mounting bracket with IV pole clamp (monitor: V1030)



3387



3388

Roll Stands

ORDERING INFORMATION

Product Codes	Description	
3387	SurgiVet® Advisor® Vital Signs Monitor Roll Stand	The Advisor® vital signs monitor roll stand has two locking casters and a ten pound base for increased stability. A mounting plate and locking device allows for the easy attachment or removal of the monitor. A utility basket provides convenient storage. Dimensions: 45" H x 23" W.
3388	SurgiVet® Tabletop Monitor Roll Stand	This roll stand is similar to the SurgiVet® Advisor® vital signs monitor roll stand, but it excludes the 10 pound base. Com-patible with V3304, V3404, V3404P, V6004, and V9004 monitors.

Infusion



3500-0600

ORDERING INFORMATION

Product Codes	Description
3500-0600-1	Medfusion® 3500 syringe pump - new
3500-0600-82	Medfusion® 3500 syringe pump - remanufactured

Medfusion® 3500 Syringe Pump

The Medfusion® 3500 syringe pump is an intuitive system designed to enhance safety while maintaining simplicity. Precise delivery of fluids and medications allows for the implementation of a wider range of treatment modalities.

- Accepts most common syringe brands
- Accommodates a wide range of syringe sizes from 1 cc up to 60 cc
- Option to use with IV fluid bag - requires special attachment
- Extensive infusion rate range from 0.01 mL/hr to 1130 mL/hr
- Unrivaled accuracy, +/- 2% excluding syringe accuracy
- Multiple delivery mode combinations:
 - mL, mg, mcg, ng
 - kg, m2
 - hr, min, day, time
- Programmable MRI indicated for use outside of 150 Gauss line with shielded magnets of 1.5 Tesla or less



MX448L60MG

Mangum™ Infusion Set

The Mangum™ micro delivery sets are designed to enable primary IV administration of metered infusion from a Medfusion® syringe pump.

- Universal vented/non-vented bag spikes are readily compatible with either bottle or bag
- Small bore tubing available for low-volume infusions
- Sets have a built-in safety feature, a 90 degree turning radius stopcock, to eliminate the possibility of free flow to the patient

ORDERING INFORMATION

Product Codes	Description
MX448FMG	Small bore detachable extension set with 0.2 micron filter, 3.0 mL volume, 74" length
MX448L60MG	Small bore detachable extension set, 2.5 mL volume, 69" length
MX5096020MG	Small bore detachable extension set for use with 1 cc syringe, 0.5 mL volume, 68" length
MX536020MGIL	Micro bore detachable extension set, 1.3 mL volume, 68" length

Temperature Management



V707000

ORDERING INFORMATION

Product Codes	Description
V707000	HOTLINE® Fluid Warmer

HOTLINE® Fluid Warmer

FEATURES

- Allows fluid or blood to be warmed to the appropriate range of 95-105° Fahrenheit
- A patented patient line with triple lumen tubing maintains a layer of 107.6° Fahrenheit solution circulating around IV line all the way to the patient
- Warm and ready to use in four minutes (from a cold start)
- Accommodates a slow drip to a bolus
- Can be mounted to an IV pole

- Remanufactured: one-year warranty
- Dimensions: 9.5" H x 8.3" W x 7.0" D
- Weight: 7.95 lbs.
- L-70 (30) replacement warming sets

ITEMS INCLUDED WITH EACH

- 1 operations manual
- 5 L-70 warming sets
- 80-04-001 repair kit includes:
 - 2- "o" rings
 - allen wrench
 - silicone lubricant



WWVEQ5000RS

ORDERING INFORMATION

Product Codes	Description
WWVEQ5000RS	Equator® Convective Warming System (refurbished)
WWVEQ5000	Equator® Convective Warming System (new)

SurgiVet® Equator® Convective Warming System

FEATURES

- Lightweight
- Compact size
- Quiet air delivery
- Can be mounted to an IV pole
- Features a digital display and push button control for four temperature settings
- Remanufactured unit: One year warranty
- New unit: Three-year warranty
- Dimensions: 11.75" H x 9.5" W x 7.5" D
- Weight: 15 lbs.
- 6014005 power supply
- 7 ft large air delivery hose (SW5-HOSE-7)
- Filter kit (F3-5000)

ITEMS INCLUDED WITH EACH

- 32491B12 1 7 ft large air delivery hose
- SWU-2001 1 Full body blanket (40" W x 80" L)
- SWU-2002 1 pediatric full body blanket (40" W x 57.5" L)
- SWU-2004 1 lower body blanket (40" W x 64" L)
- SWU-2009 1 small pediatric underbody blanket (26" W x 37" L)
- SWU-2011 1 large pediatric underbody blanket (40" W x 54" L)
- 1 operations manual

*Level 1® Snuggle Warm® convective warming blankets-pg. 30

Accessories

ORDERING INFORMATION

Product Codes	Description
SW5-HOSE-7	7 ft large air delivery hose
F3-5000	Filter kit

Level 1® Snuggle Warm® Convective Warming Blankets



SWU-2001*



SWU-2002



SWU-2003



SWU-2004*

The disposable blankets are constructed of two layers of non-woven polypropylene fabric coated with a layer of polyethylene. The blankets are designed to conform to the patient.

Sold in ten packs only.

ORDERING INFORMATION

Product Codes	Description	Size	Compatible	Units
SWU-2001*	Full body blanket	40"W x 80"L	Large hose	10/pk
SWU-2002*	Pediatric full body blanket	40"W x 57.5"L	Large hose	10/pk
SWU-2003	Upper body blanket	80"W x 40"L	Large hose	10/pk
SWU-2004*	Lower body blanket	40"W x 64"L	Large hose	10/pk
SWU-2009*	Small pediatric underbody blanket	26"W x 37"L	Large hose	10/pk
SWU-2009	Small upper body blanket	80"W x 30"L	Large hose	10/pk
SWU-2011*	Large pediatric underbody blanket	40"W x 54"L	Large hose	10/pk
SWU-2013	Adult underbody blanket	40"W x 80"L	Large hose	10/pk

*One sample blanket included with the SurgiVet® Equator® Convective Warming System



SWU-2009*



SWU-2010



SWU-2011*



SWU-2013



L-70

HOTLINE® Disposable Administration Sets

HOTLINE® Blood and Fluid Warmers are proven in millions of surgeries, resulting in warmer patients. Smiths Medical offers a full line of disposable administration sets and accessories for use with HOTLINE® Blood and Fluid Warmers.

ORDERING INFORMATION

Product Code	Description
L-70	Blood and I.V. Fluid Warming Set with Injection Port (Latex free)

Anesthesia Equipment

Common Features of All SurgiVet® CDS 9000 Small Animal Anesthesia Machines

- Pop-off valve designed to interface with an active waste gas evacuation system and maintain a passively full rebreathing bag
- Rebreathing and non-rebreathing capability
- Easy, quick change absorber canister
- Tapered 22 mm port on the inhalation/exhalation valve and bag port allow easy changing of the breathing circuit or bag
- Fresh gas port
- Adaptable to all styles of vaporizers



*Vaporizer not included

WWV9000

SurgiVet® Universal CDS 9000 Small Animal Anesthesia Machine Pole Mount

FEATURES

- Optional conversion kits available for flexibility
- Pop-off occlusion valve included
- 3 piece assembly
- 5 year warranty
- Dimensions: 53.25" H x 26" W x 26" D
- Weight: 23 lbs.

ITEMS INCLUDED WITH EACH

- 32359B3 F-Circuit with 2 liter bag
- 32359B6 modified jackson rees with 1/2 liter bag
- CO₂ absorbent refill bag (2.2 lbs)
- V7309 pop-off occlusion valve
- WWV1932 operations manual

ORDERING INFORMATION

Product Codes	Description
WWV9000	Single Flowmeter (0.2-4 LPM)
WWV9001	Dual Flowmeter (0-1,000 cc, 0.2-4 LPM)
WWV9002	N ₂ O/O ₂ (N ₂ O: 0.2-4.5 LPM, O ₂ : 0.2-4 LPM)

OPTIONAL ITEMS NOT INCLUDED

- WWV90TT tabletop conversion kit
- WWV90WM wallmount conversion kit
- WWV90MB monitor bracket



WWV9010

SurgiVet® Universal CDS 9000 Small Animal Anesthesia Machine Tabletop

FEATURES

- Optional conversion kits available for flexibility
- Pop-off occlusion valve included
- Two built-in handles for ease of mobility
- 5 year warranty
- Dimensions: 14.5" H x 14.5" W x 10.5" D
- Weight: 15 lbs.

ITEMS INCLUDED WITH EACH


- 32359B3 F-circuit with 2 liter bag
- 32359B6 modified jackson rees with 1/2 liter bag
- CO₂ absorbent refill bag (2.2 lbs)
- V7309 pop-off occlusion valve
- WWV1932 operations manual

ORDERING INFORMATION

Product Codes	Description
WWV9010	Single Flowmeter (0.2-4 LPM)
WWV9011	Dual Flowmeter (0-1,000 cc, 0.2-4 LPM)
WWV9012	N ₂ O/O ₂ (N ₂ O: 0.2-4.5 LPM, O ₂ : 0.2-4 LPM)

OPTIONAL ITEMS NOT INCLUDED

- WWV90PK pole mount conversion kit
- WWV90WM wall mount conversion kit
- WWV90MB monitor bracket

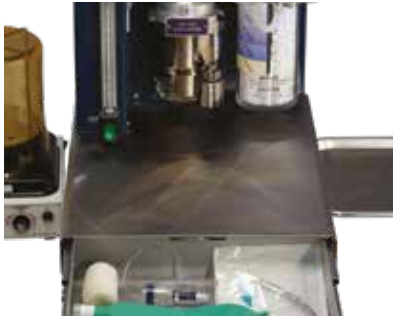

21-5500-51

SurgiVet® Universal CDS 9000 Small Animal Anesthesia Machine Conversion Kit

ORDERING INFORMATION

Product Codes	Description	For Use With
WWW90TT	Tabletop Kit	Converts pole mount machines to tabletop
WWW90WM	Wall Mount Kit	Converts machines to wall mount
WWW90PK	Pole Mount Kit	Converts tabletop machines to pole mount
WWW90MB	Monitor Kit	SurgiVet® tabletop & Advisor® vital signs monitors (V3304, V3404, V6004, V9230, V9200-V9204, V9400)
21-5500-51	Shelf and Drawer	Dimension 18" x 14.5", drawer 18" x 14.5" x 3"



V7025

ORDERING INFORMATION

Product Codes	Description
V7025	LDS 3000

SurgiVet® LDS 3000 Large Animal Anesthesia Machine

FEATURES

- Easy to clean, chip proof powder coat finish
- 22"W x 12"D x 15"H (machine only)

ITEMS INCLUDED WITH EACH

- 32387B3 2 64" black hose (2" diameter)
- V7142 stainless steel "Y" piece with luer fitting for end tidal CO₂ monitor connection
- 32346B7 30 liter bag
- 5 CO₂ absorbent refill bags (2.2 lbs)
- Operations manual

OPTIONAL ITEMS NOT INCLUDED

- V7140 foal absorber canister
- 32364B1 large animal endotracheal tube funnel
- V7114 stand alone absorber bracket mount
- V7275 DHV 1000 ventilator
- V7276 cart (Pictured)
- V7277 shelf (Pictured)
- V7138 foal circuit set
- V7026SYS Large Animal System
- Includes: V7275, V7276, V7025



V7026SYS

ORDERING INFORMATION

Product Codes	Description
V7026SYS	<p>Large Animal Anesthesia/Ventilator system consisting of:</p> <ol style="list-style-type: none"> 1. V7025 - LDS 3000 Large Animal Anesthesia Machine 2. VCT302 - Vaporizer (not included) - pg.45 3. V7275 - DHV 1000 Large Animal Ventilator - pg 43 4. V7276 - Cart (with Anesthesia Machine)

SurgiVet® Large Animal Anesthesia System

FEATURES

- Easy to clean, chip proof powder coat finish
- Dimensions: 53" H and
- 24" W diameter base with wheels
- Four wheel base for maximum stability
- Mobile

OPTIONAL ITEMS NOT INCLUDED

- V7140 foal absorber canister
- 32364B1 large animal endotracheal tube funnel
- V7114 stand alone absorber bracket mount
- V7277 shelf (Pictured)
- V7138 foal circuit set

ITEMS INCLUDED WITH EACH

- V7275 DHV 1000 ventilator
- V7276 Large animal anesthesia and ventilator cart
- 32387B3 2 64" black hose (2" diameter)
- 32387B5 46" hose (attach bag port to ventilator, 2" diameter)
- 2 ft O₂ hose with DISS female connectors on each end
- V7142 stainless steel "Y" piece with luer fitting for end tidal CO₂ monitor connection
- 32346B7 30 liter bag
- 32411B10 power pack with cord, 100-220 V, 50/60 Hz
- 32423C7 bellows
- 5 CO₂ absorbent refill bags (2.2 lbs)
- Operations manual

Common Features of All SurgiVet® Lab/Small Animal Anesthesia Machines

- Non-rebreathing system (ideal for patients 15 lbs. and under)



*Vaporizer not included

V702002

FEATURES

- Great for feline only practices or research settings
- Lightweight and portable



V703001

FEATURES

- Ideal for research
- Multiple patients can be anesthetized at the same time
- Each station can be controlled individually

SurgiVet® Tabletop Research Anesthesia Machine

ORDERING INFORMATION

Product Codes	Description
V702001	0.2-4 LPM O ₂ flowmeter
V702002	Zero-1,000 cc O ₂ flowmeter
V702003	Dual O ₂ flowmeter (0-1,000 cc, 0.2-4 LPM)
V702004	O ₂ , N ₂ O, and fail-safe (N ₂ O: 0-4.5 LPM, O ₂ : 0.2-4 LPM)
V702005	Air and O ₂ flowmeter (air: 0-5 LPM, O ₂ : 0.2-4 LPM)
V702006	Air only flowmeter (0-5 LPM)

ITEMS INCLUDED WITH EACH

- V7103 rodent circuit set
- 32359B2 avian research circuit and 1/4 liter bag

- Operations manual

SurgiVet® Multi-Station Lab Animal Anesthesia Machine

ORDERING INFORMATION

Product Codes	Description
V704001	2 station (primary flowmeter 0-3 LPM, station flowmeter 0-3 LPM)
V703501	4 station (primary flowmeter 0-5 LPM, station flowmeter 0-3 LPM)
V703001	6 station (primary flowmeter 0-8 LPM, station flowmeter 0-3 LPM)

- Each station can deliver a different percentage of anesthetic without affecting any other station
- Mobile

ITEMS INCLUDED WITH EACH

- V7103 rodent circuit set per station
- 1- 36" silicone hose per station
- Operations manual



V7015

SurgiVet® Vaporstick Anesthesia Machine

ORDERING INFORMATION

Product Codes	Description
V7015	Single O ₂ flowmeter (0.2-4 LPM)
V701501	Air and O ₂ flowmeter (0-1,000 cc, 0.2-4 LPM)
V701502	O ₂ , N ₂ O, and fail-safe (N ₂ O: 0-4.5 LPM, O ₂ : 0.2-4 LPM)
V701504	Single O ₂ flowmeter (0-1,000 cc)

FEATURES

- Great for feline only practices or research settings
- Can be used to provide supplemental oxygen for intensive care units in emergency or critical care situations
- Compact design, mobile

ITEMS INCLUDED WITH EACH

- 32359B6 modified jackson rees non-rebreathing system with 1/2 liter bag
- 32359B2 avian research circuit and 1/4 liter bag
- V7121 "Z" bracket for vaporizer
- Operations manual



V7321

V7321 Low Oxygen Pressure Alarm

ORDERING INFORMATION

Product Codes	Description
V7321	Low Oxygen Pressure Alarm

FEATURES

- Provides an audible alert when the oxygen supply is running low

- Simple DISS fittings on the included oxygen hose make the alarm adaptable to any location such as a primary oxygen supply
- Preset to alarm at 35 psi
- Equipped with a test button
- Utilizes a 9-volt battery that is included
- Dimensions: 6.25" H x 3.75" W x 1.5" D



V7320

V7320 High Patient Pressure Alarm

ORDERING INFORMATION

Product Codes	Description
V7320	High Patient Pressure Alarm

FEATURES

- Provides an audible alert when a patient circuit reaches a dangerous pressure level

- The alarm is adjustable, but is preset at 20 cm H₂O pressure
- Can be used with rebreathing and non-rebreathing patient circuits
- Equipped with a test button
- Utilizes a 9-volt battery that is included
- Dimensions: 6.1" H x 3.5" W x 1.2" D
- 32345B3 Alarm Pressure line adapter

Induction Chambers

ORDERING INFORMATION

Product Codes	Description	Size/Material
V711802	Small induction chamber	4" W x 4" H x 6" L (acrylic)
V711801	Medium induction chamber	4" W x 4" H x 12" L (acrylic)

FEATURES

- Calmly anesthetize patients in a stable container



Induction Chambers



Breathing Bags

Breathing Bags

ORDERING INFORMATION

Product Codes	Description
32346B1	1 liter non-latex
32346B5	2 liter non-latex
32346B6	3 liter non-latex
32346B8	5 liter latex
32346B9	6 liter latex
32346B2	15 liter latex
32346B7	30 liter latex

Silicone Tubing

ORDERING INFORMATION

Product Codes	Description
32372B1	1/4" inner diameter medical grade silicone tubing is sold by the foot
32372B2	1/8" inner diameter medical grade silicone tubing is sold by the foot

Universal F-Circuit

The universal F-circuit is designed to maintain humidity and prevent heat loss. The F-circuit fits almost any anesthesia machine designed to deliver rebreathing anesthesia.

*(1) flex tubing-V7133, (2) F-circuit elbow-32343B10, (3) adapter 22 mm male x female (cuff)-32343B6, (4) 2 liter bag-32346B5



32359B3

ORDERING INFORMATION

Product Codes	Length	Inner Diameter at Patient End	Outer Diameter at Patient End	Inner Diameter at Valve End	Outer Diameter at Valve End	Items Included
32359B3	40"	15 mm	22 mm	22 mm	28 mm	2 Liter Bag
32359B4	60"	15 mm	22 mm	22 mm	28 mm	-
32359B10	120"	15 mm	22 mm	22 mm	28 mm	-

Pediatric Standard "Y" Circuit

The Pediatric Standard "Y" circuit fits almost any anesthesia machine designed to deliver rebreathing anesthesia.

ORDERING INFORMATION

Product Codes	Description	Length	Breathing Bag	Elbow /Connector	Wye
490804	Pediatric, expandable	50/100"	1.0 L	Gas Sampling Elbow	Straight



32359B6

Modified Jackson Rees


ORDERING INFORMATION

Product Codes	Description
32359B6	Modified Jackson Rees

(1) adapter supply line-32343B15, (2) 1/2 liter bag, (3) 13 mm male x 19 mm female adapter-32343B27

FEATURES

- The modified jackson rees is designed for a non-rebreathing system. The control valve allows you to close the system for manual ventilation and then reopen for spontaneous breathing. The waste gas is evacuated from the circuit by an evacuation hose with a 19 mm fitting.

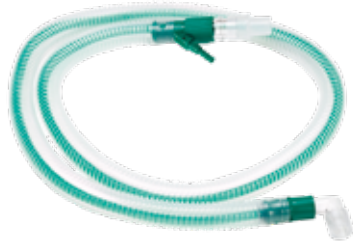


V710401

C-Prax Breathing Circuit

The C-Prax is a non-rebreathing circuit designed to maintain humidity and reduce patient heat loss.

ORDERING INFORMATION

Product Codes	Length	Inner Diameter at Patient End	Outer Diameter at Patient End	Inner Diameter at Waste Gas End	Outer Diameter at Waste Gas End	Items Included
32135B1	60"	16 mm	22 mm	15 mm	19 mm	NA

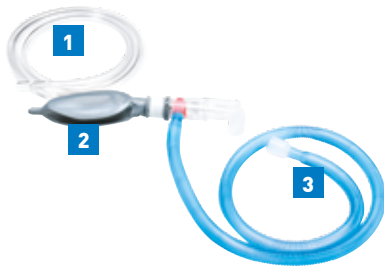

32359B2

Avian Circuit

ORDERING INFORMATION

Product Codes	Description
32359B2	Avian Circuit

(1) adapter supply line-32343B15, (2) 1/4 liter bag, (3) 13 mm male x 19 mm female adapter-32343B27

FEATURES

- The avian circuit is a non-rebreathing circuit designed for small patients where dead air space is an issue. The control valve allows you to close the manual ventilation and then reopen for spontaneous breathing. Waste gas is evacuated from the circuit by an evacuation hose with a 19 mm fitting.



V7103

Rodent Circuit Set

ORDERING INFORMATION

Product Codes	Description
V7103	Rodent Circuit Set

FEATURES

- The rodent circuit set includes a mask, diaphragm without a hole, breathing circuit, supply hose, adapters, supply line adapter, 3 ft silicone tubing, 22 mm male x 22 mm male adapter (32001B1), and a 22 mm female x 19 mm male (32343B23).



V7115

Mayo Tray

ORDERING INFORMATION

Product Codes	Description
V7115	Mayo Tray

FEATURES

- Constructed of a powder coated aluminum frame with stainless steel tray. Tray will accommodate a 1-2 inch pole. Dimensions: 15" L x 11" W

Sodasorb® LF CO₂ Absorbent

Sodasorb® LF CO₂ Absorbent

ORDERING INFORMATION

Product Codes	Description
008855	Sodasorb® LF CO ₂ absorbent plastic bottle (9.3 lbs.) (4.2 kg)
008865	Sodasorb® LF CO ₂ absorbent refill bag (2.2 lbs.) (1 kg)
008875	Sodasorb® LF CO ₂ absorbent prepack canister (2.2 lbs.) (1 kg)

FEATURES

- Permanent color change when absorbent is exhausted
- Specifically formulated to prevent the degradation of anesthesia gases that can lead to problematic by products. Exhibits high contrast color change, should dessication occur
- Long-lasting formulation
- More efficient use of anesthetic gases
- Uniform pellet design results in less dust; better absorption

Sodasorb® CO₂ Absorbent

Sodasorb® CO₂ Absorbent

ORDERING INFORMATION

Product Codes	Description
008870	Sodasorb® CO ₂ absorbent pre pak (2.2 lbs.) (1 kg)
008860	Sodasorb® CO ₂ absorbent refill bag (2.8 lbs.) (1.3 kg)
008880	Sodasorb® CO ₂ absorbent 5 gallon pail (37 lbs.) (16.8 kg)

FEATURES

- Long-lasting formulation removes CO₂ for hours while maximizing the use of anesthetic agents
- Fewer change-outs and low dust content makes cleaning easier
- Specially formulated to help reduce the risk of degradation of anesthetic agents
- Proven safety record ensures the comfort and well being of both patients and staff
- High contrast color change lets you know when it's time to replace
- May be used in conjunction with all the most commonly utilized anesthetic agents



V7306

SurgiVet® V7306 Pop-Off Valve

ORDERING INFORMATION

Product Codes	Description
V7306	Pop-Off Valve

FEATURES

- The pop-off valve maintains a constant passive volume within the anesthesia system allowing the patient to take a breath or the anesthetist to administer a breath. The rebreathing bag will stay passively full while maintaining less than 2 cm H₂O pressure. The valve will interface with a properly adjusted active waste gas evacuation system.

SurgiVet® V7309 Pop-Off Occlusion Valve



V7309

ORDERING INFORMATION

Product Codes	Description
V7309	Pop-Off Occlusion Valve

FEATURES

- The pop-off occlusion valve helps to prevent an accidental pneumothorax due to leaving the pop-off valve closed. By simply depressing a button on top, the valve temporarily blocks the gas flow so that a breath may be given to the patient. Upon release, the valve opens and no adjustment of the pop-off valve is required. A 19 mm to 22 mm adapter (32343B23) is included.

Absorbers



V7114

ORDERING INFORMATION

Product Codes	Description
V7114	Stand Alone Absorber

FEATURES

- The stand alone absorber is an add-on absorber canister for small animals weighing 15-200 lbs. that includes inhalation/exhalation valves, manometer, and a pop-off valve. This item is designed for use with the V7025 SurgiVet® LDS 3000 large animal anesthesia machine.

Non-Rebreathing Adapter



V7128

ORDERING INFORMATION

Product Codes	Description
V7128	Non-Rebreathing Adapter

FEATURES

- By placing the adapter in-line, moving from rebreathing to non-rebreathing is quick and easy.

Anesthesia Masks



V7149

ORDERING INFORMATION

Product Codes	Description	Dimensions	Replacement Diaphragm
32393B5	Small feline mask	2" x 2" x 1" diaphragm opening	32368B5
32393B3	Medium feline mask	2.25" x 3 5/8" x 1 1/8" diaphragm opening	32368B4
32393B2	Large feline mask	3.75" x 3 5/8" x 1 1/8" diaphragm opening	32368B2
32393B4	Small canine mask	3.5" x 3 5/8" x 1.25" diaphragm opening	32368B2
32393B1	Large canine mask	6" x 5.5" x 2.25" diaphragm opening	32368B1
V7149	5 mask set	Includes one of each of the above	-
V7105	Rodent mask	1.5" x 2.25"	Not included

Rodent Mask Diaphragms




Rodent Mask Diaphragms

ORDERING INFORMATION

Product Codes	Size
V710503	3/4" diaphragm opening
V710502	9/16" diaphragm opening
V710501	7/16" diaphragm opening
32247B1	Without opening (cut to desired size)

Dual Anesthesia Research Circuit

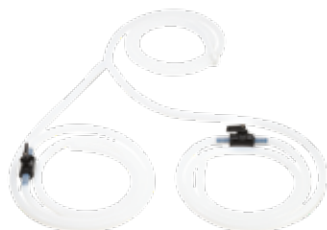


V7137

ORDERING INFORMATION

Product Codes	Description
V7137	Dual Anesthesia Research Circuit

FEATURES

- Enables gas to be directed from one station to another or both

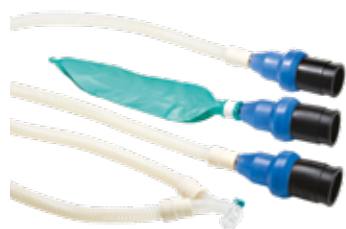

V7138

Foal Circuit Set

ORDERING INFORMATION

Product Codes	Description
V7138	Dual Anesthesia Research Circuit

ITEMS INCLUDED

- 32426B5 1-"Y" piece with luer fitting
- 32364B1 3-large animal endotracheal tube funnels
- 32364B3 3-funnel valve adapters
- 32346B6 1-3 liter bag
- 32359B5 2-60" 22 mm hoses
- 32001B1 3-22 mm x 22 mm adapters
- 32343B10 F-circuit elbow

FEATURES

- Commonly used with the LDS 3000 large animal anesthesia machine. The hoses included in the set can also be used with SurgiVet® brand small animal anesthesia machines.



V7142

"Y" Piece with Luer Fitting

ORDERING INFORMATION

Product Codes	Description
V7142	"Y" Piece with Luer Fitting

FEATURES

- Stainless steel "Y" piece for large animal machine
- Fits a 2" diameter circuit
- Luer fitting allows connection for ETCO₂ monitoring



32364B1

Large Animal Endotracheal Tube Funnel Adapter (Silicone)

ORDERING INFORMATION

Product Codes	Description
32364B1	Large animal endotracheal tube funnel adapter (silicone)

Miscellaneous Adapters

ORDERING INFORMATION

Product Codes	Dimensions	Material	Description/Usage
32001B1	22 mm male x 22 mm male	Clear acrylic	SAV 2500 ventilator
32036B1	15 mm male x 1/4" hose barb	Silver metal	Fresh gas outlet elbow
32343B15	15 mm male x 1/4" hose barb	Clear acrylic	Adapter supply line/modified jackson rees circuit
32343B18	30 mm female x 19 mm male	Purple plastic	SAV 2500 ventilator /waste gas port
32343B20	22 mm female x 17 mm female	Clear silicone	SAV 2500 ventilator
32343B23	22 mm female x 19 mm male	Translucent white plastic	Adapts 19 mm tubing to F/air canister/V7309
32343B6	22 mm male x 22 mm female (ribbed cuff)	Translucent white plastic	Universal F-circuit (connection at inhalation valve) /V7301 Lab animal evacuation unit inlets
32345B3	22 mm male x 22 mm female (ribbed cuff) with 1/8" hose barb	Translucent white plastic	High patient pressure alarm line adapter
V7311	19 mm male x 1/2" female	White plastic	AES (1/2" PVC adapter)
V7313	19 mm male x 19 mm male	Translucent white plastic	Connection of one 19 mm tubing section to another
32426B5	22 mm male x 22 mm male x 22 mm male Wye	Translucent white plastic	15 mm ID/breathing circuits

MR Conditional Equipment

Common Features of SurgiVet® CDS 9000 MR Conditional Small Animal Anesthesia Machines

All SurgiVet® MR Conditional products have been independently tested by an outside agency and have been certified MR Conditional in a static magnetic field of 3-Tesla or less.



WWV9000MR

ITEMS INCLUDED WITH EACH

- 32359B3 F-circuit with 2 liter bag
- 32359B6 modified jackson rees with 1/2 liter bag
- CO2 absorbent refill bag (2.2 lbs.)
- V7309 pop-off occlusion valve
- WWV1932 operations manual

Each MR conditional product is appropriately labeled according to ASTM standards for MR Conditional products. This allows users to easily identify products that are safe for this use.

Contact Smiths Medical for specific technical and performance characteristics for each MR Conditional product.

SurgiVet® CDS 9000 MR Conditional Small Animal Anesthesia Machines

ORDERING INFORMATION

Product Codes	Description
WWV9000MR	Single flowmeter - Pole mount (0.2-4 LPM)
WWV9001MR	Dual flowmeter - Pole mount (0-1,000 cc, 0.2-4 LPM)
WWV9010MR	Single flowmeter - Tabletop (0.2-4 LPM)
WWV9011MR	Dual flowmeter - Tabletop (0-1,000 cc, 0.2-4 LPM)

FEATURES

- Pop-off occlusion valve included
- 5 year warranty
- Dimensions: (WWV9000MR, WWV9001MR) 14.5" H x 14.5" W x 10.5"
- Weight: (WWV9000MR, WWV9001MR; 23 lbs.)
- Dimensions: (WWV9010MR, WWV9011MR) 14.5" H x 14.5" W x 10.5"
- Weight: (WWV9010MR, WWV9011MR; 15 lbs.)



VCT302MR

ORDERING INFORMATION

Product Codes	Description
VCT302MR	SurgiVet® Classic T3™ Isoflurane Funnel Fill Vaporizer
VCT3K2MR	SurgiVet® Classic T3™ Isoflurane Key Fill Vaporizer
VCT305MR	SurgiVet® Classic T3™ Sevoflurane Funnel Fill Vaporizer
VCT3K5MR	SurgiVet® Classic T3™ Sevoflurane Key Fill Vaporizer

MR Conditional Vaporizers

CLASSIC T3™ FEATURES

- Precision style, out of circuit, single agent use vaporizer
- Temperature and flow compensated
- Clearly labeled with the name of agent to be used
- Site glass provides easy viewing of liquid volume
- Available as funnel fill or key fill style
- One-piece design fill assembly to eliminate leak points
- Funnel fill dimensions: 7.25" H x 5" W x 5.5" D (18.42 cm H x 12.7 cm W x 13.97 cm D)
- Funnel fill weight: 16 lbs. (7.27 kg)
- Key fill dimensions: 7.25" H x 5" W x 6" D (18.42 cm H x 12.7 cm W x 15.24 cm D)
- Key Fill Weight: 16 lbs. (7.27 kg)
- Available in funnel fill style
- Built from all new parts based on the original Ohio style vaporizer
- Three year parts and labor warranty is included
- One-year warranty on calibration
- Annual calibration service is required by an authorized SurgiVet® vaporizer service center to maintain the warranty
- Funnel fill dimensions: 6" L x 5" W x 8" H
- Weight: 13 lbs. (5.91 kg)



32373B10

Waste Gas Collection

ORDERING INFORMATION

Product Codes	Description
32373B10	Enviro-Pure® canister 6/PKG
V7305	Enviro-Pure® charcoal filter bracket
30-4773	Waste gas evacuation tubing 19 mm
32091B1	Waste gas evacuation tubing 22 mm



V727000

ORDERING INFORMATION

Product Codes	Description
V727000	MRI 2550 Ventilator

MRI 2550 Small Animal Ventilator

FEATURES

- Pneumatically powered and controlled
- Can be used as an anesthesia ventilator or a long-term respirator
- Adjustable controls for inspiratory and expiratory time as well as inspiratory flow
- Transportable
- MRI compatible
- Dimensions: 11.75" H x 9" W x 6.5" D
- Weight: 16 lbs.

ITEMS INCLUDED WITH EACH

- 32359B7 48" 22 mm cuffed hose
- 32359B8 12" 22 mm cuffed hose
- 24" O₂ hose with DISS female connectors on each end
- 32426B3 DISS wye connector (F x M x M wye)
- 32391B1 test lung
- Operations manual

OPTIONAL ACCESSORIES

- V7265 MRI 2550 mounting bracket

MR conditional ventilators will work with non MR conditional products. Optional bellows available-pg. 35.



V727001

ORDERING INFORMATION

Product Codes	Description
V727001	Pneupac® Respiratory Ventilator

Pneupac® Respiratory Ventilator

FEATURES

- Pneumatically powered and controlled
- Designed for short or long term ventilation
- Confidently control inspiratory and expiratory times, and inspiratory flow
- Compressed gas source can be combined with room air
- Compact design
- Portable

- MRI Compatible

- Dimensions: 3.5" H x 9" W x 6.5" D
- Weight: 9 lbs.

ITEMS INCLUDED WITH EACH

- 32391B1 test lung
- 32359B8 12" 22 mm cuffed hose
- 24" O₂ hose with DISS female connectors on each end
- Operations manual

MR conditional ventilators will work with non MR conditional products. Optional bellows available-pg. 35.

Ventilators



V725000

ORDERING INFORMATION

Product Codes	Description
V725000	SAV 2500 Small Animal Ventilator

SAV 2500 Small Animal Ventilator

FEATURES

- Electronically controlled, pneumatically powered, time cycled
- Adjustable controls for inspiratory flow, inspiratory time and breaths-per-minute
- Manual pneumatic breath button in the event of power outage
- Portable
- Dimensions: 11.5" H x 9.5" W x 12" D
- Weight: 12 lbs.

OPTIONAL ACCESSORIES

- V7310 300 ml bellows
- V7312 3 liter bellows (foal)
- V7260 SAV 2500 mounting bracket

ITEMS INCLUDED WITH EACH

- 32411B16 bellows assembly
- 32359B8 12" 22 mm cuffed hose
- 32359B7 48" 22 mm cuffed hose
- 32001B1 22 mm x 22 mm adapter
- 32343B20 22 mm x 17 mm silicone adapter
- 32426B3 DISS wye connector (F x M x M wye)
- 32391B1 test lung
- 32343B18 Adapter 30 mm x 19 mm
- 32411B10 power pack with cord, 100-220 V, 50/60 Hz
- 24" O₂ hose with DISS female connectors on each end
- Operations manual



V7275

ORDERING INFORMATION

Product Codes	Description
V7275	DHV 1000

DHV 1000 Large Animal Ventilator

ITEMS INCLUDED WITH EACH

- 32387B5 46" hose (attach bag port to ventilator, 2" diameter)
- 2 ft O₂ hose with DISS female connectors on each end
- 32411B10 power pack with cord, 100-220 V, 50/60 Hz
- 32423C7 bellows
- Operations manual

OPTIONAL ACCESSORIES

- V7276 Large animal anesthesia and ventilator cart
- V7276MRI Large animal anesthesia and ventilator cart MR conditional
- V7025 Large animal anesthesia machine

V7026SYS Large Animal System consisting of:

- V7025 - LDS 3000 Large Animal Anesthesia Machine pg 31
- VCT302 - Vaporizer (not included) pg 45
- V7275 - DHV 1000 Large Animal Ventilator
- V7276 - Cart (with Anesthesia Machine)

FEATURES

- Electronically controlled, pneumatically powered with an adjustable timed cycle
- MRI model is pneumatically controlled, pneumatically powered and time cycled
- Provides controlled ventilation to anesthetized patients over 250 lbs
- Designed with descending bellows
- Mount on a cart or shelf
- Dimensions: 30" H x 16" W x 16" D
- Weight: 45 lbs.



V7310

Pediatric Bellows

ORDERING INFORMATION

Product Codes	Description
V7310	Pediatric bellows with base

FEATURES

- 300 ml bellows
- Can be used with the MRI 2550 ventilator and SAV 2500 small animal ventilator
- MR Conditional



V7312

Foal Bellows

ORDERING INFORMATION

Product Codes	Description
V7312	Foal bellows

FEATURES

- 3 liter bellows
- Can be used with the MRI 2550 ventilator and SAV 2500 small animal ventilator
- MR conditional

Vaporizers



VCT302

FEATURES

- Precision style, out of circuit, single agent use vaporizer
- Temperature and flow compensated
- Clearly labeled with the name of agent to be used

SurgiVet® Classic T3™ Vaporizer

ORDERING INFORMATION

Product Codes	Description
VCT302	Classic T3™ isoflurane funnel fill vaporizer
VCT3K2	Classic T3™ isoflurane key fill vaporizer
VCT305	Classic T3™ sevoflurane funnel fill vaporizer
VCT3K5	Classic T3™ sevoflurane key fill vaporizer

- Site glass provides easy viewing of liquid volume
- Available as funnel fill or key fill style
- One-piece design fill assembly with no solder joints eliminating leak points
- Funnel fill dimensions: 7.25" H x 5" W x 5.5" D (18.42 cm H x 12.7 cm W x 13.97 cm D)
- Funnel fill weight: 16 lbs. (7.27 kg)
- Key fill dimensions: 7.25" H x 5" W x 6" D (18.42 cm H x 12.7 cm W x 15.24 cm D)
- Key fill weight: 16 lbs. (7.27 kg)

Servicing Isoflurane and Sevoflurane Classic T3™ Vaporizers

Regular annual service is recommended to ensure proper vaporizer calibration. Cleaning involves taking the vaporizer apart and cleaning every component inside and replacing gaskets and wick when necessary. One-year warranty on calibration. To set up either service, contact Clinical Support at 1-800-258-5361.

Cleaning and Calibration

The Cleaning and Calibration Program allows the customer to have their specific vaporizer cleaned and calibration and returned to them (service takes 1-2 weeks) without a loaner being issued. Customer covers cost of shipping.

Vaporizer Exchange Program

The Vaporizer Exchange Program eliminates downtime by having a remanufactured vaporizer that has been cleaned and calibrated sent to the customer. The customer then drains their current vaporizer and ships it back to Smiths Medical using the provided UPS label and shipping container. Their vaporizer is then cleaned, calibrated and placed into the Smiths Medical stock to be sent out to another clinic requesting the exchange program.

ORDERING INFORMATION

Product Codes	Description
WWVCT302X	Classic T3™ isoflurane funnel fill vaporizer, exchange
VCT3K2-82	Classic T3™ isoflurane key fill vaporizer, exchange
WWVCT305X	Classic T3™ sevoflurane funnel fill vaporizer, exchange
VCT3K5-82	Classic T3™ sevoflurane key fill vaporizer, exchange
VCT302MR-82	Classic T3™ isoflurane funnel fill vaporizer MR conditional, exchange
VCT3K2MR-82	Classic T3™ isoflurane key fill vaporizer MR conditional, exchange
VCT305MR-82	Classic T3™ sevoflurane funnel fill vaporizer MR conditional, exchange
VCT3K5MR-82	Classic T3™ sevoflurane key fill vaporizer MR conditional, exchange

Vaporizer Accessories



V7121

ORDERING INFORMATION

Product Codes	Description
V7121	"Z" bracket allows for vaporizer to be mounted to a 1.5-2" pole
V7122	Universal vaporizer mounting bracket with end caps allows for vaporizer to be mounted to a 1.5-2" pole
32056B2	Female end cap
32056B1	Male end cap
32372B1	Silicone tubing (sold by the foot)



V7122



32056B2, 32056B1



32343B12



32343B8

Anesthetic Key Filler

ORDERING INFORMATION

Product Codes	Description
32343B12	Isoflurane
32343B17	Sevoflurane

FEATURES

- Designed to be used exclusively with key fill vaporizers
- Agent specific adapter that prevents the inadvertent mixing of agents
- Only fits the bottle of anesthetic for which it is intended

Anesthetic Anti-Spill Adapter

ORDERING INFORMATION


Product Codes	Description
32343B8	Isoflurane
32343B9	Sevoflurane

FEATURES

- Helps to minimize the loss of agent when filling a funnel style vaporizer
- Only fits the bottle of anesthetic for which it is intended

Gas Delivery Management

Hard Line Assemblies (Wall Outlets)

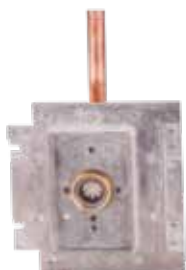

32367B4

Recessed Style Wall Outlet

ORDERING INFORMATION

Product Codes	Description
32367B4	O ₂ recessed rough in assembly
32367B3	Ohio O ₂ recessed finishing plate
32370B2	DISS O ₂ recessed finishing plate
32367B1	Ohio N ₂ O recessed finishing plate
32367B12	Recessed and surface mount repair kit

FEATURES

- Best suited for buildings being constructed (copper tubing wall inlay)
- Two parts needed to have complete assembly:
 - Recessed rough in assembly
 - Finishing plate
- Two styles of finishing plates: DISS male or ohio style female quick connect
- Unit is similar to an electrical outlet in appearance when installed



32367B3



32370B2



V7218

Oxygen Manifolds

ORDERING INFORMATION

Product Codes	Description
V7218	O ₂ manifold, 2 DISS male check inlets with a 1/4" NPT male outlet and a pressure relief valve

FEATURES

- Allows for two H-tanks to supply O₂ to facility

Soft Line Assemblies (Hose Assemblies)



V7162

Pre-Assembled Hoses

ORDERING INFORMATION

Product Codes	Description
V7162	3 ft O ₂ hose DISS female x DISS female
V7156	6 ft O ₂ hose DISS female x DISS female
V7157	8 ft O ₂ hose DISS female x DISS female
V7158	10 ft O ₂ hose DISS female x DISS female
V7163	15 ft O ₂ hose DISS female x DISS female
V7164	20 ft O ₂ hose DISS female x DISS female
V7159	6 ft O ₂ hose ohio male x DISS female
V7160	8 ft O ₂ hose ohio male x DISS female
V7161	10 ft O ₂ hose ohio male x DISS female

Hose Assemblies

- Custom hose assemblies, including ceiling hose drops
 - part number of second end plus hose assembly fee (VHASS)
- Variety of quick connects available
 - Ceiling hose drop orders may need to add Ceiling Plate (V716601) to conceal opening in ceiling
- Custom hose orders require the following: part number of one end by part number of hose with length indicated by
 - Common pre-assembled hoses available

Gas Hoses

The gas hoses are nonconductive hose and sold by the foot.

ORDERING INFORMATION

Product Codes	Description
32387B2	Green oxygen hose
32387B4	Blue nitrous hose
32387B11	Yellow air hose
32387B13	White vacuum hose
32387B12	Purple evacuation hose
32387B14	Gray carbon dioxide hose

Air Hose Connectors (Yellow)

ORDERING INFORMATION

Product Codes	Description
32370B10	DISS female x hose barb
32406B3	Ohio male quick connect x hose barb
32164B2	DISS male x 1/8" NPT male x hose barb



32370B10



32406B3

Carbon Dioxide Hose Connectors

ORDERING INFORMATION

Product Codes	Description
32164B3	DISS male x 1/8" NPT male

Evacuation Hose Connectors (Purple)

ORDERING INFORMATION

Product Codes	Description
32407B14	Chemtron male quick connect x hose barb



32407B14

Miscellaneous Hose Connectors

ORDERING INFORMATION

Product Codes	Description
32386B1	1/4" NPT female x hose barb
32386B2	1/4" NPT male x hose barb
32386B8	1/4" hose coupler, 1/4" hose barb x 1/4" hose barb
32397B1	1/4" NPT male x 1/4" NPT male 90°
32397B7	1/4" NPT male x female (elbow)
32343B31	1/4" NPT male x two 1/4" NPT female branch tee
32417B2	1/4" street T
32501B55	1/4" NPT female coupler
32032B1	1/8" NPT male x hose barb
32035B1	1/8" NPT male x hose barb 90°
32371B1	1/8" NPT male x 1/8" NPT female 45° elbow
32386B6	1/8" NPT female x hose barb
32397B2	1/8" NPT female coupler
32397B3	1/8" NPT male x female (elbow)
32397B4	1/8" NPT male x 1/8" NPT male 90°
32397B5	1/8" NPT male x hex socket plug
32417B1	1/8" street T
32427B1	Wye three 1/8" NPT female inputs
32209B1	Colder male x hose barb



32386B1



32386B2



32035B1



32397B3



32417B1

Nitrous Hose Connectors (Blue)



32370B11



32386B5

ORDERING INFORMATION

Product Codes	Description
32370B11	DISS female x hose barb
32386B5	DISS male x hose barb
32395B1	DISS male x 1/8" NPT male
32406B7	Ohio female x DISS male
32406B8	Ohio male x DISS male

Vacuum Hose Connectors



32370B12

ORDERING INFORMATION

Product Codes	Description
32370B12	DISS female x hose barb
32228B2	Ohio female quick connect x hose barb
32228B1	Ohio male quick connect x hose barb
32142B1	Chemtron male quick connect x hose barb



32228B2



32228B1



32142B1

Oxygen Hose Connectors With Barbs (Green)

ORDERING INFORMATION

Product Codes	Description
32370B5	DISS female x hose barb
32370B8	DISS male x hose barb
32407B5	Ohio female quick connect x hose barb
32407B6	Ohio male quick connect x hose barb
32407B7	Puritan female quick connect x hose barb
32407B8	Puritan male quick connect x hose barb
32363B8	Schrader female quick connect x hose barb
32407B10	Schrader male quick connect x hose barb
32407B1	Chemtron female quick connect x hose barb
32407B2	Chemtron male quick connect x hose barb
32383B2	DISS handi-tight x hose barb
V716601	Ceiling plate



32370B5



32370B8



32407B5



32407B6



32407B7



32407B8



32363B8



32407B10




32407B1



32407B2



32383B2

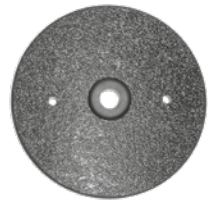

V716601

Oxygen Hose Connectors (Green)

ORDERING INFORMATION

Product Codes	Description
32034B1	DISS male x 1/8" NPT male
32164B4	DISS male x male
32370B9	DISS male x 1/4" NPT male
32370B6	DISS male demand valve x 1/8" NPT male
32370B14	DISS male check valve x 1/4" NPT male
32421B5	DISS male check valve x 1/8" NPT male
32343B14	DISS female swivel adapter x 1/8" NPT male
32386B9	DISS cap
32407B3	Ohio female x DISS female
32407B4	Ohio male x DISS female
32407B11	Ohio female x DISS male
32407B12	Ohio male x DISS male
32407B13	Ohio male x 1/8" NPT male
32363B3	Chemtron male x DISS male
32363B4	Chemtron male x DISS female
32363B6	Chemtron female x DISS female
32363B5	Chemtron female x DISS male
32363B7	Schrader male x DISS male
32406B5	Puritan male x 1/4 NPT male
32242B1	Puritan male x DISS male



32034B1



32164B4



32370B6



32421B5



32343B14



32386B9



32407B3



32407B4



32407B11



32407B12



32407B13



32363B3



32363B4



32363B6



32363B5



32242B1

E-Tank Accessories



32426B3



32426B4

ORDERING INFORMATION

Product Codes	Description
32426B3	Wye DISS female x two DISS male
32426B4	Wye ohio male x two ohio female
V7338	Wye DISS female x two DISS male with checks
V7113	Supply T dual O2 with checks (for use with two oxygen sources)

H-Tank Accessories

All single stage regulators are preset to 50-55 psi.



V7327



V7329

ORDERING INFORMATION

Product Codes	Description
V7327	O ₂ H-tank regulator
V7329	O ₂ H-tank regulator and 0-15 lpm flowmeter

Universal Double E-Tank Manifold



V7331

ORDERING INFORMATION

Product Codes	Description
V7331	Universal double E-tank manifold

FEATURES

- Internal check valves to allow one or both tanks to be turned on
- Ensures anesthesia machine maintains mobility throughout clinic or hospital
- Recommend use of V7261 with this manifold for extra stability of tanks

E-Tank Stabilizer



V7261

ORDERING INFORMATION

Product Codes	Description
V7261	E-tank stabilizer

FEATURES

- For use with V7331 to ensure stability of E-tanks
- Secures tanks in place and eliminates movement which may help reduce leaks from developing and extend the life of the V7331

E-Tank Holders and Regulator



V7330

ORDERING INFORMATION

Product Codes	Description
V7330	O ₂ E-tank regulator, double tank holder, 2 ft O ₂ hose, E-tank wrench
V7335	E-tank rings only

FEATURES

- Two sets of rings that slide over the anesthesia machine pole and secures with set screws
- One E-tank regulator, wrench and 2 ft hose included
- Tight grouping eliminates lateral swing movements
- Tanks positioned to rest on legs of base

E-Tank Accessories

All single stage regulators are preset to 50-55 psi.

ORDERING INFORMATION

Product Codes	Description
V7326	O ₂ E-tank regulator
V7332	N ₂ O E-tank regulator
V7336	Air E-tank regulator
V7328	E-tank regulator with 0-15 lpm flowmeter
32425B1	E-tank wrench
32408B7	E-tank regulator yoke
32257B2	E-tank yoke washer



V7326



32425B1



32066B1



Respiratory Flowmeters

ORDERING INFORMATION

Product Codes	Description
32066B1	Respiratory flowmeter 0-3 lpm
32375B6	Respiratory flowmeter 0-5 lpm
32375B5	Respiratory flowmeter 0-15 lpm

FEATURES

- Each has 1/8" NPT female input fitting on back and DISS male output fitting below



Flowmeter Replacement Parts

ORDERING INFORMATION

Product Codes	Description
32413B2	(1) O ₂ needle valve and knob with torque guard
32413B5	O ₂ needle valve and knob (SurgiVet® LDS 3000)
32413B6	N ₂ O needle valve and knob with torque guard
32413B7	(1) flowmeter O ₂ knob
32413B4	(2) single flowmeter lens cover
32413B1	(3) O ₂ thorpe tube 0.2-4 lpm
32413B3	Flowmeter adapter, dual threaded, rear connector
32375B14	Respiratory O ₂ flowtube housing

Humidifier System

FEATURES

- 32375B5 0-15 LPM respiratory flowmeter
- 32388B1 humidifier chamber
- Mounting apparatus
- Installation instructions

ITEMS INCLUDED WITH EACH

- Mounts to a wall or cage
- Standard Male O₂ DISS fitting to connect 50 psi oxygen supply
- Tapered hose connection on the humidifier chamber to connect cannula line or standard silicone tubing
- Oxygen hose not included

ORDERING INFORMATION

Product Codes	Description
V7144	Humidifier system



V7144

Demand Valves

ORDERING INFORMATION

Product Codes	Description
32421B11	Large animal demand valve

FEATURES

- Designed to provide 100% oxygen to breathing or nonbreathing patients
- Can be used with mask, ET tube, or tracheotomy tube
- Delivery pressure is 60 +/- 5 cm H₂O
- Large animal demand valve delivers 160 LPM

Waste Gas Management



V7225

Anesthesia Detection Badge

ORDERING INFORMATION

Product Codes	Description
V7225	Single badge
V7226	Five pack

FEATURES

- Test individuals or a facility for exposure to anesthetic gas
- The badge can be used for fifteen minutes or up to a maximum of eight hours in one day
- The prepaid test includes confidential results that are mailed back within 14 business days



V7300

SurgiVet® AES Gas Evacuation System

ORDERING INFORMATION

Product Codes	Description
V7300	110 volt system
V7300220	220 volt system

FEATURES

- Product safely removes waste gas associated with everyday inhalation anesthesia
- Waste gas is collected by passive flow to the AES, then is actively pushed up to 150 feet away
- Two 19 mm inlets and one exhaust port allows for easy installation
- Can be mounted onto a wall or anesthesia machine pole
- Dimensions: 14" H x 5" D
- Weight: 8 lbs.

ITEMS INCLUDED WITH EACH

- 32292B1 3 ft and 10 ft precut hose (13 mm diameter tubing with 13 mm x 19 mm adapters)
- V7311 1/2" PVC adapter
- Operation manual

Waste Gas Evacuation Accessories

ORDERING INFORMATION

Product Codes	Description
30-4773	19 mm tubing (per foot, blue corrugated)
32091B1	22 mm tubing (per foot)
32137B1	Evacuation tubing cap (for use with V7166, white corrugated)
32343B18	Adapter 30 mm x 19 mm (purple)
32343B23	Adapter 22 mm female x 19 mm male
V7166	Ceiling plate cover for evacuation tubing
V7313	19 mm x 19 mm male adapters



32343B18



32343B23



32373B10

SurgiVet® Enviro-Pure® Filter Canister

ORDERING INFORMATION

Product Codes	Description
WWV7318	Enviro-Pure® charcoal filter kit
32373B10	SurgiVet® 6 pack Enviro-Pure® charcoal filter canisters

ITEMS INCLUDED WITH WWV7318

- 1 Enviro-Pure® charcoal filter canister
- V7305 Enviro-Pure® charcoal filter bracket
- 32086B1 4 ft scavenger tubing

FEATURES

- Designed to absorb waste anesthetic gas
- Larger canister contains 1/3 more charcoal than comparable canisters
- Enviro-Pure® filter canister absorbs twice the agent of typical canisters
- Gram for gram, Enviro-Pure® filter is more cost effective than comparable canisters of charcoal

OPTIONAL ACCESSORIES

- V7305 Enviro-Pure® Charcoal filter bracket (for use with Enviro-Pure® charcoal filter canisters only)

Scavenger Interface Valve For Active Systems

ORDERING INFORMATION

Product Codes	Description
V7308	Scavenger interface valve for active systems

ITEMS INCLUDED WITH EACH

- 32346B5 2 liter rebreathing bag
- 1 U-bolt for mounting

FEATURES

- Designed to absorb waste anesthetic gas
- Larger canister contains 1/3 more charcoal than comparable canisters
- Enviro-Pure® filter canister absorbs twice the agent of typical canisters
- Gram for gram, Enviro-Pure® filter is more cost effective than comparable canisters of charcoal



V7301

SurgiVet® Lab Animal Evacuation System

ORDERING INFORMATION

Product Codes	Description
V7301	SurgiVet® lab animal evacuation system (110 v only)

ITEMS INCLUDED WITH EACH

- Startup quantity of activated charcoal
- Operation manual

FEATURES

- Ideal for facilities without a built-in evacuation system
- The system may be vented to the outside via a two inch outlet or activated charcoal may be used to scavenge waste gas
- Can be used with the Multi-Station Anesthesia Machine that accommodates up to 6 stations
- Dimensions: 9.75" H x 18" W x 9.75" D
- Weight: 27 lbs.



V7109

Evacuation Charcoal

ORDERING INFORMATION

Product Codes	Description
V7109	1 gallon charcoal

Rescue and Resuscitation



Resuscitation Bags

Resuscitation Bags

ORDERING INFORMATION

Product Codes	Description
32346B12	Small resuscitation (240 ml volume, under 7 kg)
32346B11	Medium resuscitation (500 ml volume, 7-30 kg)
32346B10	Large resuscitation (1600 ml volume, over 30 kg)

OPTIONAL ACCESSORIES

- WWV7267 Peep valve
- 32027B5 Manometer



V7148

Recovery Oxygen Masks

ORDERING INFORMATION

Product Codes	Description	Dimensions
32490B15	Small canine recovery mask	3" H x 3.5" W
32490B10	Large canine recovery mask	4.5" H x 5.5" W
32490B16	Feline recovery mask	2.5" H x 3.75" W
V7148	Recovery mask kit (includes one of each mask)	

FEATURES

- Dual vents enable unrestricted inhalation and exhalation
- Durable polycarbonate construction
- Supplied with 7 ft oxygen tubing
- 22 mm oxygen adapter
- Ideal for rescue operations
- Latex free

Replacement Mask Diaphragms

ORDERING INFORMATION

Product Codes	Description
32368B1	Diaphragm replacement for 32490B10
32368B2	Diaphragm replacement for 32490B15, 32490B16

VETERINARY DISPOSABLES

Airway



AT10

Non-Cuffed Endotracheal Tube (Silicone)

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Length
AT10	1	12 cm or 4.5"
AT15	1.5	12 cm or 4.5"

FEATURES

- Non-sterile
- Parallel wall design
- Autoclavable
- Radiopaque
- Commonly used in reptiles and other exotics
- Wire guide stylet



ATR25

Non-Cuffed Wire Reinforced Endotracheal Tube (Silicone)

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Tube O.D. at Shoulder	French Size	Length
ATR25	2.5	4	12	14 cm or 5.5"
ATR30	3	4.7	14	16 cm or 6"
ATR35	3.5	5.3	16	18 cm or 7"
ATR40	4	6	18	20 cm or 8"
ATR45	4.5	6.7	20	22 cm or 9"
ATR50	5	7.3	22	24 cm or 9.5"

FEATURES

- Non-sterile and transparent
- Radiopaque marker in distal tip
- Autoclavable
- Recommended to be used with malleable stylets (pg. 84)



35W060

FEATURES

- Non-sterile
- Autoclavable

Cuffed Wire Reinforced Endotracheal Tubes (Silicone)

The transparent tube has a radiopaque marker at the tip.

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Tube O.D. at Shoulder	French Size	Length	Connector (mm)
V35W25	2.5	4.0	12	14 cm or 5.5"	15
V35W30	3	4.7	14	16 cm or 6"	15
V35W35	3.5	5.3	16	18 cm or 7"	15
V35W40	4	6	18	20 cm or 8"	15
V35W50	5	7.3	22	24 cm or 9.5"	15
35W060	6	8.7	26	28 cm or 11"	15
35W065	6.5	9.4	28	28 cm or 11"	15
35W070	7	10.0	30	30 cm or 11.5"	15
35W075	7.5	10.4	32	30 cm or 11.5"	15
35W080	8	11.0	34	31 cm or 12"	15
35W085	8.5	11.8	36	31 cm or 12"	15
35W090	9	12.3	38	31 cm or 12"	15
35W095	9.5	13.3	40	31 cm or 12"	15

Cuffed Endotracheal Tubes (Silicone)

The transparent tube has a radiopaque marker at the tip.

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Tube O.D. at Shoulder	French Size	Length	Balloon Length
ET2	2	3.7	10	16 cm	2 cm
ET3	3	4.7	14	16 cm	2 cm
ET4	4	6	18	18 cm	2 cm
ET5	5	7.3	22	20 cm	3 cm
ET6	6	8.7	26	28 cm	4 cm
ET7	7	10	30	30 cm	5 cm
ET8	8	11	33	32 cm	5 cm
ET9	9	12.4	37	34 cm	5 cm
ET10	10	14.3	43	36 cm	5 cm
ET11	11	15.7	47	40 cm	5 cm
ET12	12	17	51	40 cm	5 cm
ET14	14	19	57	48 cm	5 cm
ET16	16	22	66	70 cm	7.5 cm
ET18	18	24	72	75 cm	7.5 cm
ET20	20	26.5	79	90 cm	9 cm
ET22	22	30	90	90 cm	9 cm
ET24	24	32	96	90 cm	9 cm
ET26	26	35	105	100 cm	10 cm
ET30	30	42	126	100 cm	10 cm



ET3



ET22

FEATURES

- Non-sterile
- Radiopaque
- Autoclavable
- Replaceable cuff & inflation line
- The proximal fitting of tubes up to 12 mm accepts a 15 mm adapter
- A funnel type silicone adapter is supplied with 14 mm and 16 mm tubes, & a 15 mm adapter
- 18 mm and up are supplied with a funnel type adapter only
- Medical grade silicone



DS.2940.185.15



DS.2940.150.20

Disposable Laryngoscopes

Europa AA small laryngoscope handle is a slender, lightweight handle that uses AA batteries. The disposable laryngoscope blades are stainless steel with a bright, fixed light source and available in a variety of sizes. Miller blades are straight, MacIntosh are curved.

ORDERING INFORMATION

Product Codes	Description
DS.2940.185.05	Miller #0 neonate
DS.2940.185.10	Miller #1 cat
DS.2940.185.15	Miller #2 small dog
DS.2940.185.20	Miller #3 dog
DS.2940.150.15	MacIntosh #2 cat
DS.2940.150.20	MacIntosh #3 dog
DS.2940.150.25	MacIntosh #4 large dog



292004

ORDERING INFORMATION

Product Code
292004

PressureEasy® Cuff Pressure Controller

Over inflation of the endotracheal tube cuff is a common occurrence in many veterinary practices. An over-inflated cuff can lead to localized areas of ischemia and possible pressure necrosis of the delicate tracheal tissue. Proper cuff inflation to the recommended 20-30 cm H₂O safeguards against excess pressure accumulation guarding against injury and improving patient outcomes.

FEATURES

- Guards against aspiration and tracheal damage
- Indicator window signals when cuff pressure is between 20-30 cm H₂O
- Small, lightweight design is easy to use and helps to eliminate the need for an external manometer
- Reduction in leakage of waste anesthetic gas around the endotracheal tube
- Works with all AIRCARE® endotracheal tubes
- Auto-feedback feature boosts cuff pressure to facilitate proper sealing when high pressures are used during mechanical ventilation
- Designed as single patient use to help reduce the potential for infection and disease transmission



Metal Endotracheal Tube Connectors

Endotracheal Tube Connectors

ORDERING INFORMATION

Product Codes	Tube ID (mm)
32121B3	14
32121B4	16
32121B12	18
32121B5	20
32121B6	22
32121B7	24
32121B8	26
32121B9	30
32121B10	35

FEATURES

- Manufactured to exact tolerances to ensure a safe, secure, and leak-resistant connection
- Tube connectors are designed to fit the 54mm (2") O.D. large animal wye piece with Bivona® insert



32364B1

Large Animal Endotracheal Tube Funnel Adapter (Silicone)

ORDERING INFORMATION

Product Codes	Description
32364B1	Large animal endotracheal tube funnel adapter (silicone)



RK20

FEATURES

- Non-sterile
- Autoclavable

Endotracheal Tube Replacement Accessories (Silicone)

ORDERING INFORMATION

Product Codes	Inflation Lines	Repair Cuff	Airway Adapter	Silicone Adhesive
ET2	RILS	-	-	ADH2
ET3	RILS	RK3	ETC03	ADH2
ET4	RILS	RK4	ETC04	ADH2
ET5	RILS	RK5	ETC05	ADH2
ET6	RILS	RK6	ETC06	ADH2
ET7	RILS	RK7	ETC07	ADH2
ET8	RILS	RK8	ETC08	ADH2
ET9	RILS	RK9	ETC09	ADH2
ET10	RILS	RK10	ETC10	ADH2
ET11	RILS	RK11	ETC11	ADH2
ET12	RILS	RK12	ETC12	ADH2
ET14	RILS	RK14	32121B3	ADH2
ET16	RILL	RK16	32121B4	ADH2
ET18	RILL	RK18	32121B12	ADH2
ET20	RILL	RK20	32121B5	ADH2
ET22	RILL	RK22	32121B6	ADH2
ET24	RILL	RK24	32121B7	ADH2
ET26	RILL	RK26	32121B8	ADH2
ET30	RILL	RK30	32121B9	ADH2



100/100/055

FEATURES

- Medical grade PVC is transparent and softens at body temperature to provide patient comfort
- Murphy eye to minimize potential occlusion of the tube
- Radiopaque blue line aids in confirmation of correct tube placement via x-ray
- Smooth tip helps to minimize trauma during oral intubation

AIRCARE® PVC Endotracheal Tubes

ORDERING INFORMATION

Product Codes	Size ID	Size OD	Length	Units/Box
100/100/030	3.0 mm	4.5 mm	16.5 cm	10
100/100/035	3.5 mm	5.3 mm	18.5 cm	10
100/100/040	4.0 mm	6.0 mm	21.0 cm	10
100/100/045	4.5 mm	6.5 mm	22.5 cm	10
100/100/050	5.0 mm	6.9 mm	24.5 cm	10
100/100/055	5.5 mm	7.5 mm	27.5 cm	10
100/100/060	6.0 mm	8.2 mm	28.5 cm	10
100/100/065	6.5 mm	8.7 mm	29.5 cm	10
100/100/070	7.0 mm	9.3 mm	31.0 cm	10
100/100/075	7.5 mm	10.0 mm	31.5 cm	10
100/100/080	8.0 mm	10.7 mm	33.0 cm	10
100/100/085	8.5 mm	11.3 mm	33.0 cm	10
100/100/090	9.0 mm	12.0 mm	33.0 cm	10
100/100/010	10.0 mm	13.3 mm	33.0 cm	10

Nasotracheal and Tracheostomy Tubes



FN8

Foal Nasal/Porcine Endotracheal Tube (Silicone)

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Tube OD (mm)	Length
FN7	7	10	55 cm or 22"
FN8	8	11	55 cm or 22"
FN9	9	12.3	55 cm or 22"
FN10	10	14.3	55 cm or 22"
FN11	11	15.7	55 cm or 22"
FN12	12	17	55 cm or 22"
FN14	14	19	55 cm or 22"

FEATURES

- Tubes up to 12 mm have a 15 mm airway adapter fitting
- A silicone funnel is supplied on the 14 mm tube size
- Repair balloon cuffs are available
- Replacement inflation lines are available
- Autoclavable
- Radiopaque
- Non-sterile



ENAT14

Equine Post Anesthetic Nasal Tube (Silicone)

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Tube OD (mm)	Length
ENAT14	14	19	40 cm or 16"

FEATURES

- A proximal silicone funnel fitting to avoid tube migration
- Autoclavable
- Non-sterile
- Radiopaque



V60070

Tracheostomy Tubes (Silicone)

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Length
V60050	5	6 cm or 2.5"
V60060	6	7 cm or 2.75"
V60070	7	8 cm or 3"
V60080	8	8.8 cm or 3.5"
V60090	9	9.8 cm or 4"

FEATURES

- Pre-curved silicone for a comfortable anatomical fit
- For long term or permanent tracheostomy
- Autoclavable
- Radiopaque
- Sterile

Gastroenterology

Esophageal Tubes



OTC1260

Esophagostomy Tube Sets

ORDERING INFORMATION

Product Codes	Description
OTC1260	Silicone esophagostomy catheter 12 Fr, 60 cm (24") long, autoclavable
OTC1460	Silicone esophagostomy catheter 14 Fr, 60 cm (24") long, autoclavable

FEATURES

- An open-end silicone catheter with multiple side holes used for the administration of enteral feeding solutions
- Sterile

Van Noort Esophagostomy Tube Sets

ORDERING INFORMATION

Product Codes	Tube ID (mm)
OTC1040VN	Catheter and wire guide
GTI8365ITVN	Esophagostomy introduction tube

FEATURES

- An open-ended silicone catheter with multiple side holes used for the administration of enteral solutions
- Wire guide stylet can be used to stiffen the catheter
- The introduction tube distal tip is designed for easy palpation. The slot in the tip does not allow the access needle sheath to exit the tip, but does allow the feeding catheter to do so
- Sterile



FET1423

Feline Esophagostomy Tubes (Silicone)

ORDERING INFORMATION

Product Codes	French Size	Length	Proximal Fitting
FET1423	14	23 cm or 9"	Removable
FET1938	19	38 cm or 15"	Removable

FEATURES

- A surgically placed closed-end single lumen silicone catheter with distal side holes for the administration of enteral feeding solutions
- Removable proximal fitting accepts standard Luer lock syringes
- Autoclavable catheter
- Radiopaque
- Sterile

ITEMS INCLUDED WITH EACH CATHETER

- 1 PFLLA Female Luer Lock Tapered Adapter
- 1 LLP100 Luer Lock Plug

Oral/Nasal Oxygen/Feeding Tubes



CFT520

Economy ClearView Oral/Nasal Oxygen/Feeding Tubes

ORDERING INFORMATION

Product Codes	French Size	Length
CFT3520	3.5	20 cm or 8"
CFT520	5	20 cm or 8"
CFT535	5	35 cm or 14"
CFT635	6	35 cm or 14"
CFT655	6	55 cm or 22"
CFT830	8	30 cm or 12"
CFT855	8	55 cm or 22"
CFT875	8	75 cm or 30"
CFT8120	8	120 cm or 47"
CFT1040	10	40 cm or 16"
CFT1055	10	55 cm or 22"
CFT10150	10	150 cm or 59"

FEATURES

- A closed-end, single lumen silicone catheter used for the administration of feeding solutions
- Recommended to be used with PTFE coated wire guides
- Integral cap on lightweight proximal fitting (no need for adapters or luer lock plugs)
- ClearView allows visualization of feeding solution or any occlusions
- Radiopaque
- Atraumatic
- Autoclavable
- Non-sterile

Stomach Tubes



GTIS1000

GTIS2000

FEATURES

- Reusable introduction set that enables non-endoscopic placement of gastrostomy tubes into the stomach
- Designed to accept either 16 or 20 Fr peg feeding tubes
- GTIS1000 is recommended for use with animals up to 10 kg or 22 lb
- GTIS2000 is recommended for use with animals over 10 kg or 22 lb
- Autoclavable
- Sterile



FOT1843



FMG100

Gastrostomy Tube Introduction Set

GTIS1000 ORDERING INFORMATION

Product Codes	Included Parts
GTI070TW	Stainless steel threaded wire 1 mm O.D., 70 cm (28")
GTI545IT	Introduction tube 5 mm O.D., 45 cm (18")
GTI75FA	Catheter fixation adapter, tapered
GTIN1050	Percutaneous needle 10 ga, 7 cm (2.75")

GTIS2000 ORDERING INFORMATION

Product Codes	Included Parts
GTI110TW	Stainless steel threaded wire 1 mm O.D., 110 cm (43")
GTI9585IT	Introduction tube 9.5 mm O.D., 85 cm (34")
GTI75FA	Catheter fixation adapter, tapered
GTIN1050	Percutaneous needle, 10 ga, 7 cm (2.75")

Feline Orogastric Feeding Tube

ORDERING INFORMATION

Product Codes	French Size	Length
FOT1843	18	43 cm or 17"

FEATURES

- A closed-ended single lumen silicone catheter with distal side holes used for the administration of enteral feeding solutions
- Proximal fitting accepts a tapered syringe adapter
- Autoclavable
- Radiopaque
- Non-sterile

Feline Mouth Gag

ORDERING INFORMATION

Product Codes	Description
FMG100	A reusable mouth gag designed to assist in the introduction of orogastric tubes, up to 18 Fr O.D.

FEATURES

- Non-sterile



PT16

Peg Feeding Tubes

ORDERING INFORMATION

Product Codes	French Size	Length	Mushroom Tip Diameter (mm)
PT16	16	39 cm or 16"	15
PT16XL	16	39 cm or 16"	20
PT20	20	39 cm or 16"	20
PT20XL	20	39 cm or 16"	25
PT24XL	24	39 cm or 16"	30

FEATURES

- A mushroom/pezzar style silicone catheter for the percutaneous
- administration of feeding solutions into the stomach
- Can be endoscopically-guided or used with a non-endoscopic introduction technique
- The mushroom-shaped distal tip assists in retention of the catheter in the stomach
- Catheter markings at 2 cm and 4 cm from the base of the mushroom tip assist in correct catheter placement
- Autoclavable
- Sterile
- Radiopaque

Fixation Disc (Sterile)



FD100

ORDERING INFORMATION

Product Codes	Description
FD100	Silicone fixation disc used to stabilize percutaneous drainage catheters



VSTG45

FEATURES

- Smooth-bore lumen for enhanced delivery or evacuation
- Three strategically placed fenestrations at the distal tip provide improved safety and effectiveness
- Accepts a syringe or funnel adapter
- Autoclavable
- Non-sterile

Silicone Stomach Tubes

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Tube OD (mm)	French Size	Length
VSTG30	3	5.1	15	91 cm or 36"
VSTG35	3.5	5.8	17	91 cm or 36"
VSTG45	4.5	7.3	22	91 cm or 36"
VSTG50	5	8	24	91 cm or 36"
VSTF50	5	8	24	183 cm or 72"
VSTC65	6.5	12.7	37	150 cm or 60"
VSTS65	6.5	12.7	37	300 cm or 118"
VSTM95	9.5	15.9	48	300 cm or 118"
VSTL11	11	19	57	300 cm or 118"
VSTX17	16	22	66	305 cm or 120"

Fluid Administration and Therapy



CT9530

FEATURES

- Used for the infusion of fluids into venous circulation
- Sterile

Connecting Tube

ORDERING INFORMATION

Product Codes	Description
CT95250	Flexible spiral connecting tube set 9.5 Fr, uncoiled length 250cm (98"). Rotating adapter is designed to prevent tangling of lines during infusion therapy.
CT9530	Connecting tube with male Luer and female port 9.5 Fr, 30 cm (12")
CTU14030	Connecting tube with male Luer and female port (accepts 3/4" hose barb), 14 Fr, 30 cm (12"), commonly used with heimlich one-way valve (HV100)



LACTEC3

Bag to Bag Connectors

ORDERING INFORMATION

Product Codes	Description
LACTEC3	Double-ended spiked connecting tube, 24 Fr (8 mm O.D.), 40cm long with two non-vented spikes and G-clamp

FEATURES

- Allows a second fluid bag to be connected to an existing fluid bag and delivery line
- Sterile

Wide Bore Infusion Set (No Chamber)



LACTEC1

ORDERING INFORMATION

Product Codes	Description
LACTEC1	Single, wide bore tubing with spike, 24 Fr (8 mm O.D.), 200 cm (79") length tubing with rotating cuff male Luer lock

FEATURES

- Single spike set for rapid fluid administration with Luer lock connector
- Sterile

Stainless Steel Spike Adapter




VEFSPK

ORDERING INFORMATION

Product Codes	Description
VEFSPK	Stainless steel spike adapter for high flow fluid infusions I.D. 23, O.D. 25, Spike 0.6 cm

Subcutaneous Catheter Sets

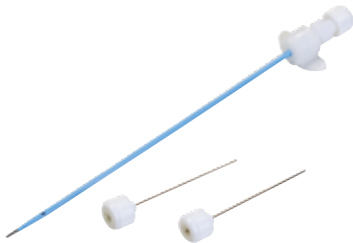

FSQ712

ORDERING INFORMATION

Product Codes	French Size	Length
FSQ712	7	12 cm or 5"
FSQ718	7	18 cm or 7"
FSQ725	7	25 cm or 10"

FEATURES

- Multiple fenestrations catheter introducer allows for easy placement
- Short stylet reinforces the catheter when not in use
- Cap prevents contamination
- Can be used long term (up to 1 year)
- Sterile

Peripheral IV Catheters



405620

Jelco® IV Catheters

ORDERING INFORMATION

Product Codes	Size	Length (in)
405320	24 ga	¾"
405020	22 ga	1"
405720	20 ga	1"
405620	20 ga	1 ¼"
405920	20 ga	1 ¾"
405520	18 ga	1 ¼"
405420	18 ga	1 ¾"
404220	16 ga	1 ¼"
404820	14 ga	1 ¼"

The Jelco® peripheral IV catheter features FEP polymer construction, "J" point needle design, flashback visualization and an easy to handle length.



505020

OPTIVA® IV Catheters

ORDERING INFORMATION

Product Codes	Size	Length (in)
505320	24 ga	¾"
505020	22 ga	1"
505620	20 ga	1 ¼"
505720	20 ga	1"
505520	18 ga	1 ¼"
504220	16 ga	2"
504820	14 ga	2"

The OPTIVA® peripheral IV catheter has an electro-polished "V" point super sharp needle. It features flashback visualization, smooth cannula deployment, easy to handle length and Ocrilon® polyurethane tubing construction that softens to body temperature and resists kinking.

Blood Collection



972306

The Saf-T Wing® blood collection and infusion set handles delicate venous blood draws, utilizing a safety feature activated with a simple one- or two- handed technique. It is available in three needle sizes and two tubing lengths with or without a pre-attached Saf-T Holder® blood tube holder.

Saf-T Wing® Blood Collection & Infusion Set

ORDERING INFORMATION

Product Codes	Description
972106	21 ga x 3/4" winged needle with 6" tubing
972112	21 ga x 3/4" winged needle with 12" tubing
972306	23 ga x 3/4" winged needle with 6" tubing
972312	23 ga x 3/4" winged needle with 12" tubing
972506	25 ga x 3/4" winged needle with 6" tubing
972512	25 ga x 3/4" winged needle with 12" tubing
982106	21 ga x 3/4" winged needle with 6" tubing with Saf-T Holder® blood tube holder
982112	21 ga x 3/4" winged needle with 12" tubing with Saf-T Holder® blood tube holder
982306	23 ga x 3/4" winged needle with 6" tubing with Saf-T Holder® blood tube holder
982312	23 ga x 3/4" winged needle with 12" tubing with Saf-T Holder® blood tube holder
982506	25 ga x 3/4" winged needle with 6" tubing with Saf-T Holder® blood tube holder
982512	25 ga x 3/4" winged needle with 12" tubing with Saf-T Holder® blood tube holder

Injection Caps and T-connector Extension Sets



SM5000

Injection Caps

Injection caps are used to cap off and protect an IV catheter while maintaining easy access to the vessel.

ORDERING INFORMATION

Product Codes	Description
MX492	0.1 mL volume, 1" length, latex-free, male Luer lock
SM5000	0.1 mL volume, 1" length, latex-free, needle-free access port with Luer lock
MX491	Dual-function Luer lock cap for male and female ports (dead-ender)



MX491



MX492

Gravity Fluid Administration Sets



SM872

Gravity Fluid Administration Sets

Intravenous fluid administration sets feature SmartSite® needleless access valves, a non-tortuous fluid path, roller clamp for fluid rate adjustment or complete occlusion and a swivel male Luer lock connection.

ORDERING INFORMATION

Product Codes	Description
SM872	20 drop/mL, 103 in. length, vented, 2 SmartSite® needleless valves

Pressure Infusion



MX4810

C-Fusor® Pressure Infusor

The crystal clear material allows for maximum visibility of fluid levels. It features an accurate and easy to read pressure gauge, quick, secure fastening and a large squeeze bulb for rapid and reliable pressure control.

ORDERING INFORMATION

Product Codes	Description
MX4805	C-Fusor® 500 mL pressure infusor, 1 each
MX4810	C-Fusor® 1000 mL pressure infusor, 1 each

FEATURES

- Secure fastening
- Durable and easy to clean
- Allows for even pressure around the fluid container
- Three-way stopcock prevents air leaks
- 1000 mL and 500 mL sizes



MX4710P1

Clear-Cuff® Pressure Infusor

The Clear-Cuff® Pressure Infusor has a thick and durable polyurethane construction and provides good visibility of the fluid bag and fluid level.

ORDERING INFORMATION

Product Codes	Description
MX4705	500 mL, 1 each
MX4710	1000 mL, 1 each

FEATURES

- The built-in hook allows the fluid bag to be replaced without removing the infusor from the IV pole
- Consistent fluid flow for effective fluid management
- The extra-large squeeze bulb provides for fast inflation of the infusor. The HI-FLO™ stopcock allows the infusor to be deflated almost instantly without squeezing the bag

T-connector Extension Sets

T-connector extension sets are used to provide easy access to IV catheters and the convenient connection of IV fluid lines.

ORDERING INFORMATION

Product Codes	Description
MX459SL	0.3 mL priming volume, 5" length, small bore tubing, injection site, slide clamp, swivel male Luer lock connector
SM5008	0.3 mL priming volume, 5" length, small bore tubing, needle-free access port slide clamp, swivel male Luer lock adapter
MX457L	Small bore extension with swivel male Luer lock, Non-DEHP formulation, latex-free



MX459SL



SM5008



MX457L

Extension Sets

ORDERING INFORMATION

Product Codes	Replacement Parts
MX454FL	2.3 mL priming volume, 21" length, standard bore, no access ports , pinch clamp
537235PS	1.3 mL priming volume, 72" length, small bore light resistant tubing, no access ports, pinch clamp
SM5010	Standard bore extension set, pinch clamp, SmartSite® needle free valve, slide swivel Luer lock, Non-DEHP formulation, latex free



537235PS



SM5010

Needles



VC1409

Vet-Core™ Biopsy Needles

ORDERING INFORMATION

Product Codes	Gauge	Length (in)
VC1409	14	9 cm or 3.5"
VC1415	14	15 cm or 6"
VC1609	16	9 cm or 3.5"
VC1615	16	15 cm or 6"
VC1809	18	9 cm or 3.5"

FEATURES

- Used for soft tissue core biopsy
- Facilitates one-handed operation during procedural use
- Etched tip enhances visualization during ultrasound imaging
- Precise, rapid firing spring mechanism provides intact core tissue sample
- Bevelled point stylet permits easy penetration into the specimen with less trauma to surrounding tissue
- 20 mm specimen notch
- Sterile



ESJN1525


Needles with Peel-Off Sheaths

ORDERING INFORMATION

Product Codes	Gauge	Sheaths	Needle
NPOS1570	15	5	1
NPOS1770	17	5	1

FEATURES

- PTFE Peel-off Sheaths
- Radiopaque needle and sheath
- Sterile

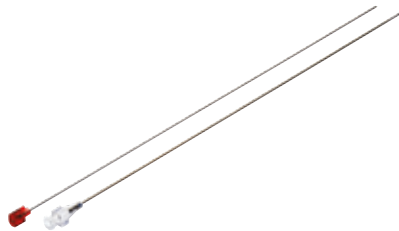

ESJN1525

Equine Sacral Joint Needle

ORDERING INFORMATION

Product Codes	Gauge	Length (in)
ESJN1525	15	25 cm or 10"

FEATURES

- Needle used for injection or aspiration of the sacral joint
- The inner stylet matches the bevel tip of the outer needle
- Sterile
- Echogenic tip on outer cannula

Reproduction



VEUF15

Equine Uterine Flushing Catheter

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Tube OD (mm)	French size	Length
VEUF15	8	12	36	150 cm or 4' 11"
VEUF80	8	12	36	80 cm or 2' 7"

FEATURES

- Used for flushing and irrigation, administration of medications, embryo collection and transfer
- Autoclavable (silicone tube only)
- Sterile



VEFD16

Equine Fluid Delivery System

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Tube OD (mm)	French size	Length
VEFD16	8	13	39	160 cm or 5' 3"

FEATURES

- Includes a 5' 3" reusable silicone tube with a shut off clamp and bell shaped bottle adapter with a medication port/vent
- For use with the Equine Uterine Flushing Catheter
- Autoclavable (silicone tube only)
- Sterile

Urology



TTM30110E

*TTM30110E is commonly used in ferrets and not radiopaque

Slippery Sam™ Tomcat Urethral Catheters

ORDERING INFORMATION

Product Codes	French Size	Length	Side Holes	End Hole Design
TTM35110E	3.5	11 cm or 4"	None	Open
TTM35140E	3.5	14 cm or 6"	None	Open
TTM35180E	3.5	18 cm or 7"	None	Open
TTM30110E*	3.0	11 cm or 4"	None	Open
TTM3511	3.5	11 cm or 4"	Yes	Closed
TTM3514	3.5	14 cm or 6"	Yes	Closed

FEATURES

- Polytetrafluoroethylene (PTFE) catheter with endhole or distal side holes for the drainage or administration of fluids
- Highly lubricious catheter shaft material helps ensure ease of catheter placement
- Low profile silicone hub enables syringe attachment for flushing or connection to a closed collection system
- Suture holes in the silicone hub allow securing of the catheter to the prepuce
- Catheter is rigid enough that a stylet is not required
- Radiopaque
- Sterile

Tomcat Urethral Catheters (Silicone)

ORDERING INFORMATION

Product Codes	French Size	Length	Side Holes	End Hole Design
TM35	3.5	12 cm or 5"	Yes	Closed
TM350E	3.5	12 cm or 5"	Yes	Open

FEATURES

- Distal side holes or an endhole for drainage or administration of fluids
- Wire guide stylet supplied to assist introduction of the catheter
- Proximal fitting enables syringe attachment, and suture holes in the base plate allow suturing to the prepuce
- Open or closed tip
- Sterile

Little Herbert® Luer Lock Adapter

ORDERING INFORMATION

Product Codes	Description
TCLLA	Standard female Luer lock fitting with barbed insertion end

FEATURES

- Allows for firm connection of drainage lines to urinary catheters without interfering with the comfort factor
- Sterile



TCLLA



FC555

FEATURES

- Closed-end silicone catheter with distal balloon cuff used for intermittent drainage of urine from the bladder or contrast studies of urogenital tract
- Ribbed balloon provides even inflation and aids in retaining the catheter inside the bladder
- Recommended to only be used with PTFE coated wire guides (pg. 84)
- Sterile
- Radiopaque
- Atraumatic
- Autoclavable
- Color coded for easy size reference

Economy ClearView Foley Catheters (Silicone)

ORDERING INFORMATION

Product Codes	French Size	Length	Balloon Size	Colour
CFC530	5	30 cm or 12"	1 cc	Pink
CFC555	5	55 cm or 22"	1 cc	Pink
CFC630	6	30 cm or 12"	1.5 cc	Brown
CFC655	6	55 cm or 22"	1.5 cc	Brown
CFC830	8	30 cm or 12"	3 cc	Blue
CFC855	8	55 cm or 22"	3 cc	Blue
CFC1030	10	30 cm or 12"	3 cc	Black
CFC1055	10	55 cm or 22"	3 cc	Black
CFC1255	12	55 cm or 22"	5 cc	White
CFC1455	14	55 cm or 22"	30 cc	Green
CFC1655	16	55 cm or 22"	30 cc	Orange
CFC1855	18	55 cm or 22"	30 cc	Red
CFC2065	10	65 cm or 26"	30 cc	Yellow
CFC2265	22	65 cm or 26"	30 cc	Purple

*Use only sterile water to inflate balloon



FC555

*Use only sterile water to inflate balloon

FEATURES

- Closed-end silicone catheter with distal balloon cuff used for intermittent drainage of urine from the bladder or contrast studies of urogenital tract
- Balloon aids in retaining the catheter inside the bladder
- Recommended to be used with wire guide stiffening stylets
- Sterile
- Radiopaque
- Atraumatic
- Autoclavable
- Color coded for easy size reference

Premium Foley Catheters (Silicone)

ORDERING INFORMATION

Product Codes	French Size	Length	Balloon Size	Colour
FC530	5	30 cm or 12"	2 cc	Pink
FC555	5	55 cm or 22"	2 cc	Pink
FC630	6	30 cm or 12"	2 cc	Grey
FC655	6	55 cm or 22"	2 cc	Grey
FC830	8	30 cm or 12"	5 cc	Blue
FC855	8	55 cm or 22"	5 cc	Blue
FC1030	10	30 cm or 12"	5 cc	Black
FC1055	10	55 cm or 22"	5 cc	Black
FC1230	12	30 cm or 12"	10 cc	White
FC1255	12	55 cm or 22"	10 cc	White
FC1455	14	55 cm or 22"	10 cc	Green
FC1655	16	55 cm or 22"	20 cc	Orange
FC1855	18	55 cm or 22"	20 cc	Red
FC2065	20	65 cm or 26"	30 cc	Yellow
FC2265	22	65 cm or 26"	30 cc	Purple
SFC2875 (Stallion)	28	75 cm or 30"	50 cc	NA
SFC28100 (Stallion)	28	100 cm or 39"	50 cc	NA

*Use only sterile water to inflate balloon

Central Venous Catheters



CVS301J

CVS501J

FEATURES

- Used for venous pressure monitoring, blood sampling and administration of drugs and fluids
- Intended to be introduced using the Seldinger introduction technique
- Sterile



CVP4010

Central Venous Seldinger Catheter Sets

CVS301J AND CVS401J ORDERING INFORMATION

Product Codes	Feline Description
CVS301J	Set consists of 3 Fr polyurethane catheter, 8 cm (3") long, 4 Fr vessel dilator, 30 cm (12") "J" wire guide and 21 ga, 4 cm (1.6") long needle
CVS401J	Set consists of 4 Fr polyurethane catheter, 10 cm (4") long, 5 Fr vessel dilator, 50 cm (20") "J" wire guide and 20 ga, 4 cm (1.6") long needle

CVS501J ORDERING INFORMATION

Product Codes	Canine Description
CVS501J	Set consists of 5 Fr polyurethane catheter, 15 cm (6") long, 6 Fr vessel dilator, 50 cm (20") "J" wire guide and 18 ga, 7 cm (2.8") long needle

Central Venous Peel-off sheath catheter sets

ORDERING INFORMATION

Product Codes	Description
CVP308	Set consists of 3 Fr polyethylene catheter 8 cm (3") long, matched flexible obturator, 19 ga, 7 cm (2.8") long peel-off sheath needle and 10 ml syringe
CVP4010	Set consists of 4 Fr polyurethane catheter 10 cm (4") long, matched flexible obturator, 17 ga, 7 cm (2.8") long peel-off sheath needle and 10 mL syringe
CVP5015	Set consists of 5 Fr polyurethane catheter 15 cm (6") long, matched flexible obturator, 15 ga, 7 cm (2.8") long peel-off sheath needle and 10 mL syringe

FEATURES

- Used for various pressure monitoring, blood sampling and administration of drugs and fluids
- Central venous catheter introduced using a peel-off sheathed meedle
- The peel-off sheath pre-loaded on the access meedle allows percutaneous introduction of the venous catheter, following direct venipuncture and removal of the needle cannula
- After positioning the catheter, the sheath is peeled away
- Sterile

Wound Management

Chest Drains



CT20

*Product Code PFLLA is recommended for use with CT12 to CT20

*Product Code PFLAL is recommended for use with CT30

Chest Drainage Tubes

ORDERING INFORMATION

Product Codes	Tube ID (mm)	Tube OD (mm)	French Size	Length	Side Holes Over Distal of Catheter
CT12	2.3	4	12	42 cm or 16.5"	5 cm or 2"
CT14	2.9	4.7	14	42 cm or 16.5"	7 cm or 3"
CT20	4.0	6.6	20	42 cm or 16.5"	10 cm or 4"
CT30	7.0	10	30	42 cm or 16.5"	12 cm or 5"

FEATURES

- Open-end, single lumen silicone catheter with large multiple side holes over the distal portion of the catheter
- Markings at 2 and 4 cm from the last hole aid in the correct positioning of the catheter
- Recommended to be used with wire guide stiffening stylets
- Catheter can be used for pneumo-, chylo-, pyo-, or hemothorax
- Autoclavable
- Sterile
- Radiopaque

Multi-Purpose Drainage Tube*



APFT1455

ORDERING INFORMATION

Product Codes	French Size	Length
APFT1455	14	55 cm or 22"

*Introduction tubes on page 85

FEATURES

- Used as extension tubes in dwelling drainage catheters or pressure lines
- Made of polyvinyl chloride
- Sterile



TPT1020

Pneumothorax Sets

ORDERING INFORMATION

Product Codes	French Size	Catheter Length	Introduction Needle Stylet Length
TPT815	8.3	35 cm or 14"	15 cm or 6"
TPT1020	10	40 cm or 16"	20 cm or 8"

FEATURES

- Used to treat a spontaneous or tension pneumothorax
- The Luer lock fitting on the catheter can be connected to suction apparatus or a heimlich valve
- Polyurethane catheter
- Sterile
- Radiopaque



HV100

Heimlich One-Way Valve


ORDERING INFORMATION

Product Codes

HV100

FEATURES

- Sterile
- Connecting tube included

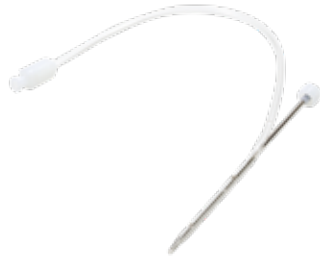

TPTL1230

Thoracic Drainage Catheter

ORDERING INFORMATION

Product Codes	French Size	Catheter Length	Introduction Needle Stylet Length
TPTL1230	12	30 cm or 12"	12 cm or 5"
TPTL1630	16	30 cm or 12"	12 cm or 5"

FEATURES

- Multi-purpose drain suitable for thoracic or abdominal drainage
- Multiple large side ports facilitate drainage of pus, haem or chyle
- The Luer lock fitting on the catheter can be connected to a suction apparatus
- Polyvinyl with trocar
- Radiopaque
- Sterile



TDC1230

Two-Part Drainage Catheter Sets


ORDERING INFORMATION

Product Codes	French Size	Catheter Length
TDC1030	10.2	30 cm or 12"
TDC1230	12	30 cm or 12"

FEATURES

- Multiple large side ports facilitate drainage of pus, heme or chyle
- The Luer lock fitting on the catheter can be connected to a suction apparatus
- Polyurethane catheter with trocar
- Radiopaque
- Sterile

Drainage Catheters and Tubes

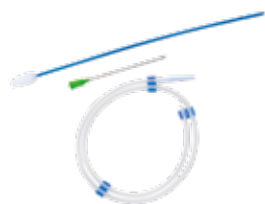

PCS911

Pericardiocentesis Catheter Set

ORDERING INFORMATION

Product Codes	Description
PCS911	Set includes 8.2 Fr, 20 cm (8") long catheter with six distal side holes, an 18 ga, 7 cm (3") long introduction needle and a 0.038" diameter, 60 cm (23") long wire guides

FEATURES

- Used to drain fluid from the pericardial sac
- FEP (fluorinated ethylene propylene copolymer) catheter
- Sterile

Pericardiocentesis Needle & Catheter

ORDERING INFORMATION

Product Codes	French Size	Length	Gauge	Needle Length
PCS1415	8.2	15 cm or 6"	14	18 cm or 7"

FEATURES

- Used to drain fluid from the pericardial sac
- FEP (fluorinated ethylene propylene copolymer) catheter
- Sterile



PCS1415

Broncho-Alveolar Lavage Catheter (Silicone)

ORDERING INFORMATION

Product Codes	French Size	Tube ID (mm)	Tube OD (mm)	Needle Length
BAL240	30	3.0	11	240 cm or 96"
BAL300	30	3.0	11	300 cm or 120"

FEATURES

- Open end catheter used for the lavage of selected segments of the lung fields
- The distal balloon can be inflated to prevent retrograde fluid flow
- The proximal two-way fitting allows simultaneous connection for both irrigation and aspiration
- Repair balloon cuffs and silicone adhesive are available (RK7, ADH2)
- Replacement inflation lines are available (RILS)
- Autoclavable
- Sterile



BAL240

The three-way plastic stopcock provided is not autoclavable and is intended for one time use (Product Code: STC3WSLIP)



CBAL5190

Canine Broncho-Alveolar Lavage Catheter

ORDERING INFORMATION

Product Codes	French Size	Length
CBAL5190	5	190 cm or 75"
CBAL6125	6	125 cm or 50"

FEATURES

- Used for the lavage of selected segments of the lung fields
- Polytetrafluoroethylene (PTFE) catheter with endhole and female luer at the opposite end
- Sterile
- Non-autoclavable (best to use ethylene oxide sterilization)
- Radiopaque



EFC6250

Guttural Pouch Endoscopic Flushing Catheter

ORDERING INFORMATION

Product Codes	French Size	Length	Distal Tip Markings
EFC5250	5	250 cm or 98"	Yes
EFC6250	6	250 cm or 98"	Yes
EFC6330	6	330 cm or 130"	Yes
EFC7250	7	250 cm or 98"	Yes
EFC7350	7	350 cm or 138"	Yes

FEATURES

- Used for lung lavage or introduction into the guttural pouch to flush out pouch debris
- Visual indicators at 2.5, 5, and 7.5 cm from the distal tip, aid in introduction
- PTFE catheter
- Sterile



PLS100

Peritoneal Lavage Catheter

ORDERING INFORMATION

Product Codes	Description
PLS100	Set includes 8.2 Fr, 20 cm (8") long catheter with six distal side holes, 18 ga, 7 cm (3") long introduction needle and 0.038" diameter, 60 cm (23") long wire guide

FEATURES

- Used to infuse and withdraw rinsing solutions in body cavities in cases of blunt trauma and therapeutic use
- Tip is slightly curved at the distal third of the catheter
- FEP (fluorinated ethylene propylene copolymer)
- Sterile



Continuous Suction Bulbs

Continuous Suction Bulbs

ORDERING INFORMATION

Product Codes	CC
GSB100	100
GSB150	150
GSB400	400

FEATURES

- Allows short-term suction to be applied to body cavities enabling the drainage of either air or fluid
- One-way valve and luer lock fitting
- Provides active suction
- Bulb is ideal for use in small animals
- Side tab allows the device to be anchored to the body by either bandage or tape
- Clear silicone allows you to note the characteristics of the fluid that is being collected
- Sterile

Wound Drains



FCWD25


Fenestrated Catheter Wound Drain

ORDERING INFORMATION

Product Codes	French Size	Length
FCWD25	8	25 cm or 10"

FEATURES

- Used to drain fluid from the pericardial sac
- FEP (fluorinated ethylene propylene copolymer) catheter
- Sterile

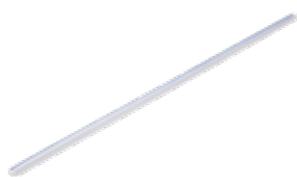

MSP630

Modified Penrose Drains (Silicone)

ORDERING INFORMATION

Product Codes	Diameter (mm)	Length
MSP630	6	30 cm or 12"
MSP1230	12	30 cm or 12"

FEATURES

- Used for post-operative drainage
- Internal ribs increase surface area, enhancing drainage by capillary action
- Transparent
- Radiopaque
- Sterile



PD0830

Penrose Drains (Silicone)

ORDERING INFORMATION

Product Codes	Diameter (mm)	Length
PD0830	8	30 cm or 12"
PD1230	12	30 cm or 12"

FEATURES

- Used for surgical wound drainage
- Autoclavable
- Radiopaque
- Sterile

Closed Wound Vacuum Drains (Silicone)

ORDERING INFORMATION

Product Codes	Drainage Tube Length	Width (mm)	Multiple Fenestrations	Rounded Connection Tube
CWVD0720	20 cm or 8"	7	Yes	60 cm or 24"
CWVD1320	20 cm or 8"	13	Yes	60 cm or 24"

FEATURES

- Used for post-operative drainage
- May be used in conjunction with closed wound vacuum devices
- Radiopaque
- Sterile
- Can be used with Continuous Suction Bulb

CWVD0720



Flexi-Drain Multi-Purpose Wound Drain (Silicone)

ORDERING INFORMATION

Product Codes	Lumen Diameter (mm)	Drainage Tube Length	Joined Catheter Lumens
WD1240	3	40 cm or 16"	12

FEATURES

- May be trimmed or used as one piece for surgical wound drainage
- Drain can be split along its length or trimmed to suit individual requirements
- Radiopaque
- Autoclavable
- Sterile

WD1240



Consumable Product Accessories

Stopcocks (Sterile)



OWS100

ORDERING INFORMATION

Product Codes	Description
OWS100	One-way plastic stopcock, female Luer lock to male Luer lock
STC3W	Three-way plastic stopcock, two female Luer locks to male Luer lock with right side arm
STC1W	One-way plastic stopcock, female Luer slip to male Luer slip
STC3WSLIP	Three-way plastic stopcock, two female Luer slips to one male Luer slip



STC3W



STC3WSLIP



STC1W

Multi-Purpose Tubing Adapters (Sterile)

ORDERING INFORMATION

Product Codes	Description
PFLLA	Female Luer lock to tapered catheter end Fitting accepts 10-20 Fr I.D.
PFLLAL	Female Luer lock to large tapered catheter end Fitting accepts 20-40 Fr I.D.
PMLLAL	Male Luer lock to large tapered catheter end Fitting accepts 20-40 Fr I.D.
P412	Double-ended tapered connector Fitting accepts 4-12 mm I.D.
P79	Double-ended tapered connector Fitting accepts 7-9 mm I.D.



PFLLA



PFLLAL



PMLLAL



P412



P79



LLP100

Luer Lock Plug (Sterile)

ORDERING INFORMATION

Product Codes	Description
LLP100	Luer lock plug to occlude female Luer lock



CLAMPWS



CLAMPWL

Clamps (Sterile)

ORDERING INFORMATION

Product Codes	Description
CLAMPWS	Clamp to occlude drainage catheter tubing for use with tubing up to 5 mm O.D.
CLAMPWL	Large clamp to occlude drainage catheter tubing for use with tubing 12.7-16 mm O.D.



ADH2

Silicone Adhesive (Non-Sterile)

ORDERING INFORMATION

Product Codes	Description
ADH2	Single 2.8 oz tube of silicone adhesive recommended for repair of balloon cuff and inflation lines



FD100

Fixation Disc (Sterile)

ORDERING INFORMATION

Product Codes	Description
FD100	Silicone fixation disc used to stabilize percutaneous drainage catheters

Wire Guides, Stylets, Introduction Tubes



WWVSTYLSM

Endotracheal Tube Stylet With Stopper

ORDERING INFORMATION

Product Codes	Fits (mm)
WWVSTYLSM	2.5-5.5
WWVSTYLLG	5.5-9.5

FEATURES

- Stainless steel endotracheal tube stylet with silicone stopper
- Non-sterile
- Autoclavable
- Malleable



TCGW2170


ClearView PTFE Coated Wire Guide

ORDERING INFORMATION

Product Codes	Outer Diameter (in)	Length	For use with CFC Feeding Tubes (pg 68)
TCGW2170	0.021"	70 cm or 28"	5 Fr & 6 Fr
TCGW3570	0.035"	70 cm or 28"	8 Fr
TCGW3870	0.038"	70 cm or 28"	10 Fr
TCGW38180	0.038"	180 cm or 71"	10 Fr

FEATURES

- Sterile
- Autoclavable

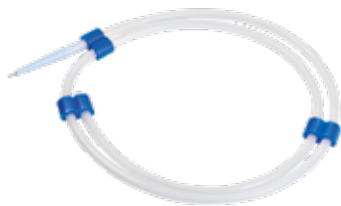

WG2560

Wire Guide Stiffening Stylets

ORDERING INFORMATION

Product Codes	Outer Diameter (in)	Length	For use with FC Foley Catheters (pg 68)
WG1770	0.017"	70 cm or 27.5"	5 Fr & 6 Fr
WG2560	0.025"	60 cm or 24"	6 Fr
WG3860	0.038"	60 cm or 24"	8 Fr
WG4560	0.045"	60 cm or 24"	10 Fr & 12 Fr

FEATURES

- Sterile
- Autoclavable



MSS5240

FEATURES

- Sterile
- Autoclavable

Metal Stiffening Stylets

ORDERING INFORMATION

Product Codes	Outer Diameter (mm)	Length	For use with CT
MSS5240	1.3	40 cm or 16"	12 Fr
MSS9040	2.3	40 cm or 16"	14 Fr
MSS18745	4.7	45 cm or 18"	30 Fr
MSS11945	3.2	40 cm or 16"	20 Fr



GTI070TW

Stainless Steel Threaded Wire Guides

ORDERING INFORMATION

Product Codes	Outer Diameter (mm)	Length
GTI070TW	1	70 cm or 28"
GTI110TW	1	110 cm or 43"

FEATURES

- Non-sterile
- Autoclavable



GTI545IT

Introduction Tubes

ORDERING INFORMATION

Product Codes	Outer Diameter (mm)	Length
GTI545IT	5	45 cm or 18"
GTI9545IT	9.5	45 cm or 18"
GTI9585IT	9.5	85 cm or 34"

FEATURES

- Non-sterile
- Autoclavable

Index

ITEM NUMBERS

SYMBOLS

21-0460-51	19	1616	25	32001B1	40	32346B5	35
21-0461-51	19	1616AU	25	32032B1	49	32346B6	35
21-0462-51	19	1616EU	25	32034B1	52	32346B7	35
21-0463-51	19	1616UK	25	32035B1	49	32346B8	35
21-0464-01	26	3005A	26	32036B1	40	32346B9	35
21-0465-10	26	3311	18	32056B1	46	32346B10	59
21-0466-50	26	3311L	18	32056B2	46	32346B11	59
21-0467-20	26	3387	27	32066B1	54	32346B12	59
21-0468-01	25	3388	27	32091B1	42 56	32359B2	36
21-5500-51	32	3393	26	32121B3	63	32359B3	35
30-4773	42 56	3409	25	32121B4	63	32359B4	35
35W060	61	3500-0600-1	28 30	32121B5	63	32359B6	36
35W065	61	3500-0600-82	28	32121B6	63	32359B10	35
35W070	61	5093	22	32121B7	63	32363B3	52
35W075	61	6012	25	32121B8	63	32363B4	52
35W080	61	6111	25	32121B9	63	32363B5	52
35W085	61	8044	23	32121B10	63	32363B6	52
35W090	61	8061	22	32121B12	63	32363B7	52
35W095	61	8075	23	32135B1	36	32363B8	51
100/100/010	64	8208	23	32137B1	56	32364B1	40 63
100/100/030	64	8211	23	32142B1	50	32367B1	47
100/100/035	64	8217	22	32164B2	48	32367B3	47
100/100/040	64	8223	22	32164B3	49	32367B4	47
100/100/045	64	8404	25	32164B4	52	32367B12	47
100/100/050	64	8408	25	32209B1	49	32368B1	59
100/100/055	64	8409	26	32228B1	50	32368B2	59
100/100/060	64	8411	24	32228B2	50	32370B2	47
100/100/065	64	8416	25	32242B1	52	32370B5	51
100/100/070	64	008855	37	32247B1	39	32370B6	52
100/100/075	64	008860	37	32257B2	54	32370B8	51
100/100/080	64	008865	37	32343B6	40	32370B9	52
100/100/085	64	008870	37	32343B8	46	32370B10	48
100/100/090	64	008875	37	32343B9	46	32370B11	50
1151	24	008880	37	32343B12	46	32370B12	50
1178N	23	9048	23	32343B14	52	32370B14	52
1179	23	9193T	23	32343B15	40	32371B1	49
1187	23	31543B1	20 22	32343B17	46	32372B1	35 46
1502	22	31543B2	20	32343B18	40 56	32372B2	35
1613	25	31543B3	20	32343B20	40	32373B10	42 57
1613AU	25	31543B4	20	32343B23	40 56	32375B5	54
1613EU	25	31544B1	20	32343B31	49	32375B6	54
1613UK	25	31544B2	20	32345B3	40	32375B14	55
		31544B3	20	32346B1	35	32383B2	51
		31545B1	20	32346B2	35	32386B1	49

32386B2	49	32413B4	55	982106	73	CFT875	67
32386B5	50	32413B5	55	982112	73	CFT1040	67
32386B6	49	32413B6	55	982306	73	CFT1055	67
32386B8	49	32413B7	55	982312	73	CFT3520	67
32386B9	52	32417B1	49	982506	73	CFT8120	67
32387B2	48	32417B2	49	982512	73	CFT10150	67
32387B4	48	32421B5	52			CLAMPWL	88
32387B11	48	32421B11	55	A		CLAMPWS	88
32387B12	48	32425B1	54	ADH2	88	CT12	81
32387B13	48	32426B3	53	APFT1455	81	CT14	81
32387B14	48	32426B4	53	AT10	60	CT20	81
32393B1	38	32426B5	40	AT15	60	CT30	81
32393B2	38	32427B1	49	ATR25	60	CT9530	70
32393B3	38	32490B10	59	ATR30	60	CT95250	70
32393B4	38	32490B15	59	ATR35	60	CTU14030	70
32393B5	38	32490B16	59	ATR40	60	CVP308	80
32395B1	50	32491B2	19	ATR45	60	CVP4010	80
32397B1	49	32491B3	19	ATR50	60	CVP5015	80
32397B2	49	32501B55	49			CVS301J	80
32397B3	49	292004	62	B		CVS401J	80
32397B4	49	404220	72	BAL240	83	CVS501J	80
32397B5	49	404820	72	BAL300	83	CWVD0720	86
32397B7	49	405020	72			CWVD1320	86
32406B3	48	405320	72	C			
32406B5	52	405420	72	CBAL5190	84	D	
32406B7	50	405520	72	CBAL6125	84	DS.2940.150.15	62
32406B8	50	405620	72	CFC530	79	DS.2940.150.20	62
32407B1	51	405720	72	CFC555	79	DS.2940.150.25	62
32407B2	51	405920	72	CFC630	79	DS.2940.185.05	62
32407B3	52	490804	35	CFC655	79	DS.2940.185.10	62
32407B4	52	504220	72	CFC830	79	DS.2940.185.15	62
32407B5	51	504820	72	CFC855	79	DS.2940.185.20	62
32407B6	51	505020	72	CFC1030	79		
32407B7	51	505320	72	CFC1055	79	E	
32407B8	51	505520	72	CFC1255	79	EFC5250	84
32407B10	51	505620	72	CFC1455	79	EFC6250	84
32407B11	52	505720	72	CFC1655	79	EFC6330	84
32407B12	52	537235PS	75	CFC1855	79	EFC7250	84
32407B13	52	972106	73	CFC2065	79	EFC7350	84
32407B14	49	972112	73	CFC2265	79	ENAT14	65
32408B7	54	972306	73	CFC520	67	ESJN1525	76
32413B1	55	972312	73	CFT520	67	ET2	61 64
32413B2	55	972506	73	CFT535	67	ET3	61 64
32413B3	55	972512	73	CFT635	67	ET4	61 64
				CFT655	67	ET5	61 64
				CFT830	67		
				CFT855	67		

ET6	61 64	FOT1843	68	MX4810	74	SWU-2002	30
ET7	61 64	FSQ712	71	MX9501T	21	SWU-2003	30
ET8	61 64	FSQ718	71	MX9543	21	SWU-2004	30
ET9	61 64	FSQ725	71	MX95114	21	SWU-2009	30
ET10	61 64			MX536020MGIL	28	SWU-2011	30
ET11	61 64	G		MX5096020MG	28	SWU-2013	30
ET12	61 64	GSB100	85				
ET14	61 64	GSB150	85	N		T	
ET16	61 64	GSB400	85	NPOS1570	76	TCGW2170	89
ET18	61 64	GTI070TW	68 90	NPOS1770	76	TCGW3570	89
ET20	61 64	GTI75FA	68			TCGW3870	89
ET22	61 64	GTI110TW	68 90	O		TCGW38180	89
ET24	61 64	GTI545IT	68 90	OTC1040VN	66	TCLLA	78
ET26	61 64	GTI8365ITVN	66	OTC1260	66	TDC1030	82
ET30	61 64	GTI9545IT	90	OTC1460	66	TDC1230	82
		GTI9585IT	68 90	OWS100	87	TM35	78
F		GTIN1050	68			TM350E	78
F3-5000	29			P		TPT815	81
FC530	79	H		P79	87	TPT1020	81
FC555	79	HV100	82	P412	87	TPTL1230	82
FC630	79	L		PCS911	83	TPTL1630	82
FC655	79	LACTEC1	71	PCS1415	83	TTM30110E	78
FC830	79	LACTEC3	70	PD0830	85	TTM3511	78
FC855	79	LLP100	88	PD1230	85	TTM35110E	78
FC1030	79	M		PFLA	87	TTM3514	78
FC1055	79	MSP630	85	PFLAL	87	TTM35140E	78
FC1230	79	MSP1230	85	PLS100	84	TTM35180E	78
FC1255	79	MSS5240	90	PT16	69		
FC1455	79	MSS9040	90	PT16XL	69	V	
FC1655	79	MSS11945	90	PT20	69	V21-6319-0100-01	17
FC1855	79	MSS18745	90	PT20XL	69	V35W25	61
FC2065	79	MX240	22	PT24XL	69	V35W30	61
FC2265	79	MX448FMG	28			V35W35	61
FCWD25	85	MX448L60MG	28	S		V35W40	61
FD100	69 88	MX454FL	75	SFC2875 (Stallion)	79	V35W50	61
FET1423	66	MX457L	75	SFC28100 (Stallion)	79	V1030	12
FET1938	66	MX459SL	75	SM872	74	V1175	23
FMG100	68	MX491	73	SM5000	73	V1180	24
FN7	65	MX492	73	SM5008	75	V1181	24
FN8	65	MX950	21	SM5010	75	V1182	24
FN9	65	MX950SCNT	21	STC1W	87	V1186	23
FN10	65	MX4705	74	STC3W	87	V1700	18
FN11	65	MX4710	74	STC3WSLIP	87	V1703	18
FN12	65	MX4805	74	SW5-HOSE-7	29	V1707	18
FN14	65			SWU-2001	30	V1709	18

V1720	18	V7225	56	V90040	15	VCT302MR	41
V3078	18	V7226	56	V90041	15	VCT302MR-82	45
V3110TL	19	V7261	53	V90041LF	15	VCT305	45
V3110TLL	19	V7275	43	V90043	15	VCT305MR	41
V3110TLS	19	V7300	56	V701501	34	VCT305MR-82	45
V3304	13	V7301	57	V701502	34	VEFD16	77
V3395	13	V7305	42	V701504	34	VEFSPK	71
V3396	25	V7306	37	V702001	33	VEUF15	77
V3397	25	V7308	57	V702002	33	VEUF80	77
V3406FL	19	V7309	38	V702003	33	VSTC65	70
V3406FLS	19	V7310	44	V702004	33	VSTF50	70
V3413	20	V7311	40	V702005	33	VSTG30	70
V3417	20	V7312	44	V702006	33	VSTG35	70
V3422	20	V7313	40 56	V703001	33	VSTG45	70
V6148SA	20	V7320	34	V703501	33	VSTG50	70
V6401	21	V7321	34	V704001	33	VSTL11	70
V6402	21	V7326	54	V706000	19	VSTM95	70
V7015	34	V7327	53	V706001	19	VSTS65	70
V7025	32	V7328	54	V706004	19	VSTX17	70
V7026SYS	32	V7329	53	V707000	29		
V7103	36	V7330	54	V710501	39	W	
V7105	38	V7331	53	V710502	39	WD1240	86
V7109	58	V7332	54	V710503	39	WG1770	89
V7113	53	V7335	54	V711801	34	WG2560	89
V7114	38	V7336	54	V711802	34	WG3860	89
V7115	36	V7338	53	V716601	51	WG4560	89
V7121	46	V8400D	14	V725000	43	WW1025OUS	24
V7122	46	V8401	14	V727000	42	WW1025SYS	24
V7128	38	V9200	16 17	V727001	42	WW1026SYS	24
V7142	39	V9200IBP/Temp	16	V7300220	56	WW1027	25
V7144	55	V9200P	16	VC1409	76	WW1028OUS	24
V7148	59	V9201	16	VC1415	76	WW1028SYS	24
V7149	38	V9201P	16	VC1609	76	WW1090	25
V7156	48	V9203	16	VC1615	76	WW1095	25
V7157	48	V9204	16	VC1809	76	WW1095OUS	25
V7158	48	V60040	14	VCT3K2	45	WW1096OUS	25
V7159	48	V60041	14	VCT3K2-82	45	WW1096SYS	25
V7160	48	V60045	14	VCT3K2MR	41	WW1098	26
V7161	48	V60046	14	VCT3K2MR-82	45	WW1098C	26
V7162	48	V60050	65	VCT3K5	45	WW1100	24
V7163	48	V60060	65	VCT3K5-82	45	WW1140	23
V7164	48	V60070	65	VCT3K5MR	41	WW3005	26
V7166	56	V60080	65	VCT3K5MR-82	45	WW3356	26
V7218	47	V60090	65	VCT302	45	WWW90MB	32
						WWW90PK	32

WWV90TT	32
WWV90WM	32
WWV3418	20
WWV7318	57
WWV9000	31
WWV9000MR	41
WWV9001	31
WWV9001MR	41
WWV9002	31
WWV9010	31
WWV9010MR	41
WWV9011	31
WWV9011MR	41
WWV9012	31
WWVCT302X	45
WWVCT305X	45
WWVEQ5000	29
WWVEQ5000RS	29
WWVSTYLLG	89
WWVSTYLSM	89

SMITHS MEDICAL TECHNICAL SERVICES & SUPPORT

CONFIDENCE WITH EXPERT FACTORY SERVICE

Smiths Medical Technical Services & Support offers convenient options to maximize uptime of critical equipment while controlling costs. You deserve the peace of mind that comes with high quality service and support. Service personnel at Smiths Medical focus on the needs of the healthcare institutions, so that care providers can focus on the needs of the patients.

Out of Warranty Services (per return)

Flat Fee Repair Pricing Program

Return a device that is out of warranty for factory service at a flat fee service rate. Get accurate factory repairs with improved service turnarounds, and reduced purchase order administration.

- Factory Diagnostics
- Manufacturer's Specification Parts
- Factory Trained Labor
- Full Functional Testing
- Free Return Ground Shipping
- Manufacturer's Service Report

NOTE:

- Payment confirmation must be received to obtain return authorization
- Smiths Medical will maintain parts for service up to five years after the end-of-life of these products
- Accessories are excluded from repair programs
- Programs may be discontinued at any time
- Pricing subject to change

Equipment Warranty Extension Program (per device, per year)

Manufacturer's Warranty Extension Program

Have peace of mind that your devices are covered by the manufacturer.

- Extends Factory Warranty Options for up to five years
- Includes parts and labor to restore device to factory operating specifications
- Free Return Ground Shipping
- Free loaner device
- Devices out of warranty are eligible after a one time Preventative Maintenance Inspection and service to bring device within specification

Accidental Damage Coverage

Add Accidental Damage Coverage on top of your Extended Warranty to protect your assets and maintain coverage of the fleet of devices by the factory. Coverage includes:

- Drops
- Unintended Damage*
- Contamination Ingress
- All parts labor for recommended repair*

* Intentional damage is excluded. Devices with missing, re-used or counterfeit components are ineligible and will be individually quoted. Devices damaged beyond economical repair can be replaced at the owner's expense, and the customer will be notified.

Contact your Account Manager for a pricing plan

Notes

Notes

**SurgiVet® Customer and Clinical Support line: 1-800-258-5361, prompt 3
Or email: veterinary.support@smiths-medical.com**

For more information visit our website at www.surgivet.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical ASD, Inc.
6000 Nathan Lane North
Minneapolis, MN 55442, USA
Tel: 1-214-618-0218
Toll-Free USA: 1-800-258-5361

MPAUC-1630

**Rx
ONLY**

smiths medical

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the indications, contraindications, warnings and precautions. AIRCARE, SACETT, Flo-Mist, Thera-Mist, PressureEasy, Slippery Sam, Little Herbert, Jelco, OPTIVA, ProPort, LockIt Plus, EchoGlo, Pulsator, ProBloc, Portex, Ocrilon, SmartSite, SECUR SITE, SlideLock, CATH-SHIELD, PORT-A-CATH, Point-Lok, GRIPPER PLUS, SurgiVet, Advisor, Transtar, Sodasorb, StackValve, Medfusion, Mangum, Pneupac, Classic T3, Enviro-Pure, HOTLINE, Equator and the Smiths Medical and SurgiVet design mark are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. ©2017 Smiths Medical. All rights reserved. VE195802EN-112017